
# NORMBAU

PASSION FOR CARE

**Cavere® Care**  
**Inox Care®**  
**Nylon Care 300**

- DE** Montage- und  
Gebrauchsanleitung
- EN** Fixing instruction and  
directions for use
- FR** Instructions de montage et  
d'utilisation
- NL** Montage- en  
gebruiksaanwijzing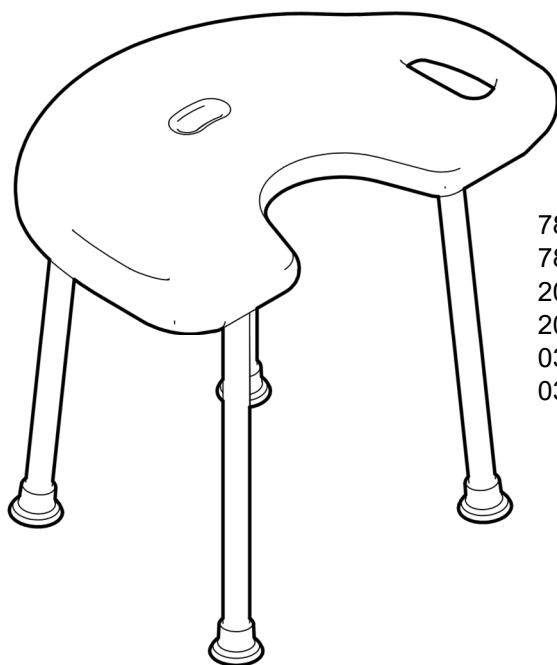

7844 345  
7844 346  
2053 450  
2053 460  
0300 345  
0300 346

Weitere Informationen  
Further information  
Plus d'informations  
Meer informatie  
[www.shop.normbau.de](http://www.shop.normbau.de)

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### **Intended use**

NORMBAU products are designed for use in wet rooms, at home, in care facilities and hospitals. They offer support to people with or without physical limitations during showering.

### **Disclaimer of liability**

NORMBAU assumes no liability for personal injury or property damage that arises or has arisen as a result of improper assembly, use or cleaning, in particular in disregard of the warning and safety instructions listed in these instructions. According to the current regulation, the warranty is two years.

Under the intended conditions of use and in compliance with the guidelines for maintenance and cleaning, the intended lifespan of the article is five years.

### **Warning notices**

There is no toxic hazard from the material used by NORMBAU. Tested according to DIN EN ISO 10993-5:2009. Product surfaces can become very hot when being exposed to external heat sources.

#### **Risk of injury**

If the max permissible user weight is exceeded, it can lead to permanent damage to the product. In this case, the product must be tested before further use!

### **Safety instructions**

Installation is to be carried out by a specialist.

Observe the information on the max. loading permitted!

Following installation, the instructions must be made available to the user.

In case of questions, more information, uncertainties or if damage occurs, contact the NORMBAU support service +49 (0) 7843 704 0 or visit us at [www.normbau.de](http://www.normbau.de).

### **Load**

Max. loading permitted of listed products: 150kg

### **Maintenance**

NORMBAU products are not maintenance-free! The products are to be tested monthly by qualified specialists for proper function.

### **Care and cleaning instructions**

#### **Risk of injury**

Improper cleaning can cause damage to the surface and lead to injuries.

## Cavere Care | Nylon Care

Powder-coated products from NORMBAU have a smooth, easy-to-clean surface. In order to maintain the decorative design and color of the coating of our products for years, cleaning and care must be carried out professionally. Slightly adhering dirt can be removed with a sponge and water to which a neutral cleaning agent (e.g. dishwashing liquid) is added. Immediately after each cleaning process, rinse with pure, cold water. A chamois or microfiber cloth removes the remaining wiping marks or drops of water. Abrasive or strong acidic/alkaline cleaning agents should never be used. With metallic coatings, heavy soiling should be cleaned as soon as possible, since dried and old dirt can often only be removed abrasively and with the risk of damage to the powder coating or a change in color. Common wiping disinfectants can be used.

## Inox Care

Stainless steel products from NORMBAU have a smooth, satin finish, easy-to-clean surface. The material has proven itself both with frequent external loads and when used in a corrosive environment. However, it should be noted that stainless steel can also rust without care and by the action of aggressive agents. Damage to the passive protective layer created by oxygen must be avoided. Dirty parts can be cleaned with a soft, damp cloth or sponge. If heavily soiled, we recommend a commercially available detergent. Common wiping disinfectants can be used. Cleaners with hydrochloric acid, citric acid and acetic acid (tile cleaner) and cleaning sponges made of steel wool are not suitable for use on stainless steel.

## Disposal

Disposal of the product and packaging is subject to the relevant national regulations and must be disposed of in the prescribed manner.

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NORMBAU GmbH Schwarzwaldstr. 15 77871 Renchen   DE
**NORMBAU GmbH**