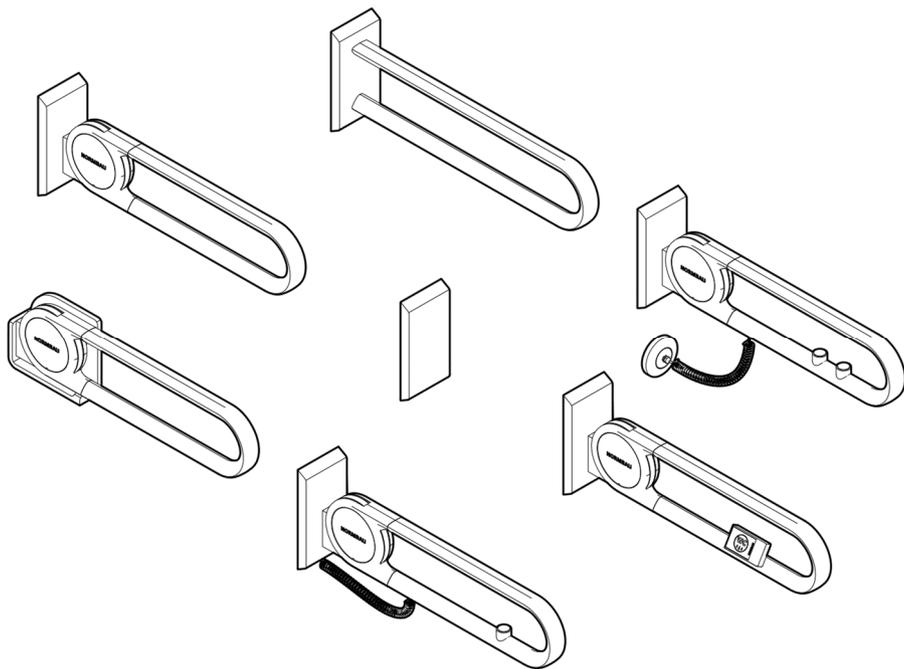
NORMBAU

PASSION FOR CARE

Cavere[®] Care
Cavere[®] Care Chrome

- DE** Montage- und
Gebrauchsanleitung
- EN** Fixing instruction and
directions for use
- FR** Instructions de montage et
d'utilisation
- NL** Montage- en
gebruiksaanwijzing

Weitere Informationen
Further information
Plus d'informations
Meer informatie
extranet.normbau.de

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Intended use

Lift-up support rails and wall support rails are suitable as aids for sanitary equipment in the washing or toilet area. The technical handle systems offer support and holding options and support independent and safe movements for people with physical disabilities.

Disclaimer of liability

NORMBAU assumes no liability for personal injury or property damage that arises or has arisen as a result of improper assembly, use or cleaning, in particular in disregard of the warning and safety instructions listed in these instructions.

Serious incidents that occur in connection with the product must be reported to the manufacturer and the competent authority of the country in which the user is established.

Warranty

The warranty period is 2 years in accordance with current regulations.

If the intended operating conditions and the maintenance and cleaning guidelines are respected, the expected service life is at least 10 years. The product has been tested and validated based on this service life.

Warning notices

There is no toxic hazard from the material used by NORMBAU. Tested according to DIN EN ISO 10993-5:2009.

Risk of injury

The product surfaces can become very hot when exposed to external heat sources, e.g. direct sunlight or contact with hot water.

Risk of injury

If the max. loading permitted is exceeded, it can lead to permanent damage to the product. In this case, the product must be tested before further use!

Risk of injury

When folding down the seat, ensure that there is no part of the body under the product.

Safety instructions

Installation is to be carried out by a specialist and only with NORMBAU mounting sets or suitable fastening material. Before installation, the wall structure must be checked to determine the fastening material. Observe the information on the max.

loading permitted and the instructions for mounting! Following installation, the instructions must be made available to the user.

In case of questions, mor information, uncertainties or if damage occurs, contact the NORMBAU support service +49 (0) 7843 704 0 or visit us at www.normbau.de.

Load

Max. loading permitted of listed products: 100kg

With max. length and max. loading permitted, the max. force per fastening is 3kN.

Maintenance

NORMBAU products are not maintenance-free! The products are to be tested monthly by qualified specialists for proper function. In the event of defects (e.g. insufficient braking force), the assembly instructions must be observed.

Maintenance and cleaning

Risk of injury

Improper cleaning can cause damage to the surface and lead to injuries.

Cavere Care

Powder-coated products from NORMBAU have a smooth, easy-to-clean surface. In order to maintain the decorative design and color of the coating of our products for years, cleaning and care must be carried out professionally. Slightly adhering dirt can be removed with a sponge and water to which a neutral cleaning agent (e.g. dishwashing liquid) is added. Immediately after each cleaning process, rinse with pure, cold water. A chamois or microfiber cloth removes the remaining wiping marks or drops of water. Abrasive or strong acidic/alkaline cleaning agents should never be used. With metallic coatings, heavy soiling should be cleaned as soon as possible, since dried and old dirt can often only be removed abrasively and with the risk of damage to the powder coating or a change in color. Common wiping disinfectants can be used.

Cavere Care Chrome

Abrasive sponges and scouring agents should not be used to clean our Cavere Chrome products. We also advise against the use of cleaners containing solvent or acid, limescale removers, vinegar and cleaning agents containing acetic acid. The chrome surface can be cleaned using a little soap on a damp cloth. Limescale stains can be avoided by drying the tap after every use. If limescale deposits still build up, you can remove these using an environmentally friendly cleaning agent. The appropriate cleaning agent should only be left on for a short time before being rinsed thoroughly with water. In areas with hard water, it may be necessary to clean

the chrome surface at specific intervals. Please observe the following points when cleaning:

- Residues from toiletries such as shampoo, shower gel or liquid soap should be removed quickly as these can leave permanent stains.
- Harsh cleaning methods and aggressive cleaning agents can damage surfaces and even penetrate the inside of the products.
- Do not apply spray cleaners directly to the chrome surface. Spray the cleaning agent onto a soft cloth and use this to wipe the surface instead.
- Use clean water to rinse the chrome surface thoroughly after cleaning.
- Do not use any cleaning agents that contain hydrochloric acids, formic acids, chlorine bleaching or acetic acids, as these can cause considerable damage.
- Be sure to treat limescale stains and other types of soiling correctly. Any damage caused as a result of improper treatment shall not be covered under warranty.

More information about
maintenance and cleaning

Disposal

Disposal of the product and packaging is subject to the relevant national regulations and must be disposed of in the prescribed manner.

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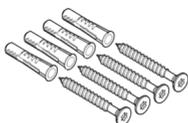
GB Pre-wall installations, lightweight walls with a backing made of at least 20 mm thick multi-layer glued wood. Pre-drill according to the screw.

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FR Maçonnerie en pierre pleine

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7470 040 / 050

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GB Masonry made of perforated stone

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



GB Pre-wall installations with steel mounting plate or metal stud wall, on lightweight walls made of plasterboard, chipboard, hardboard

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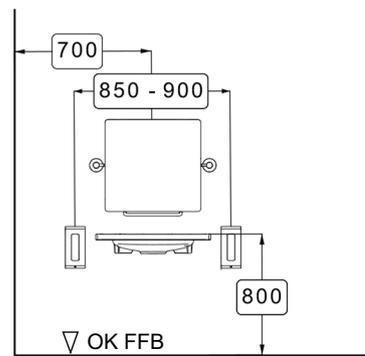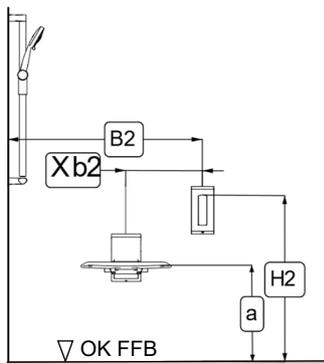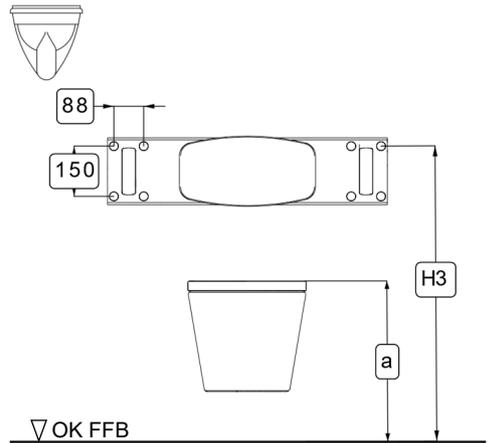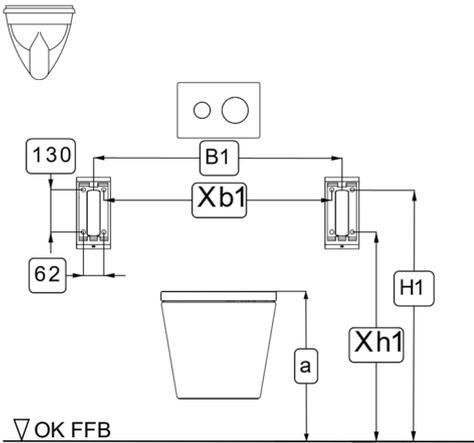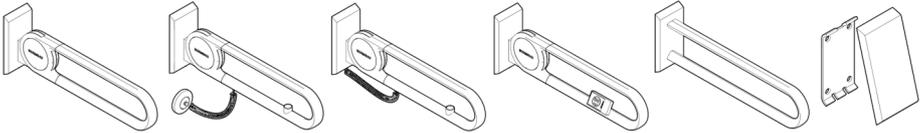
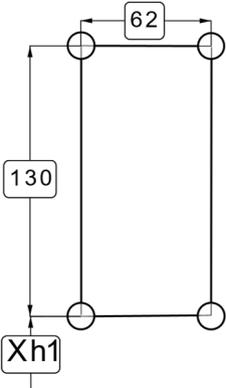
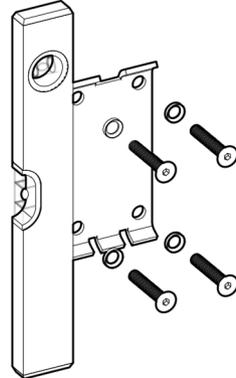
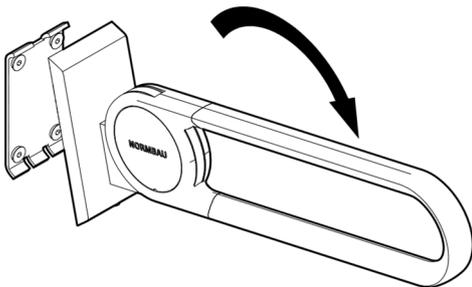
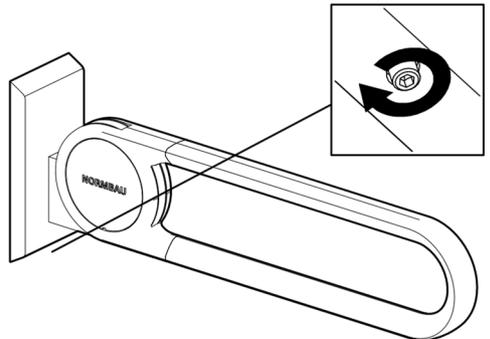
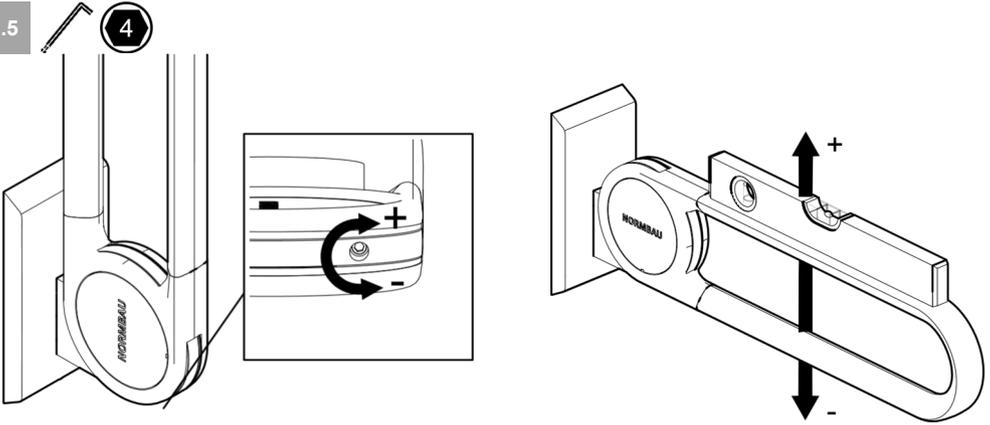
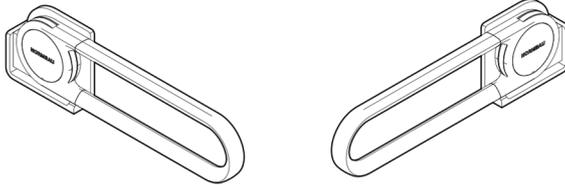
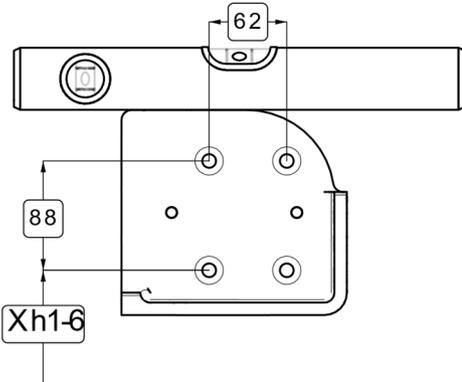
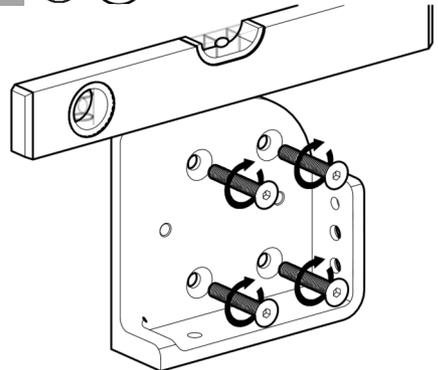
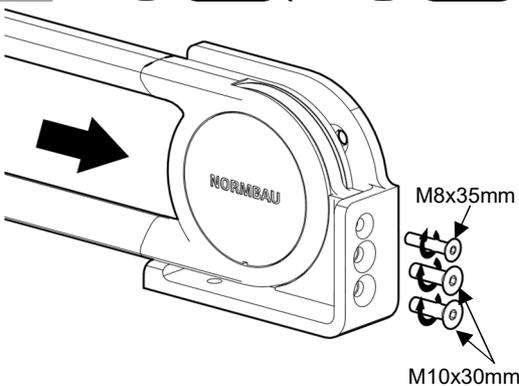
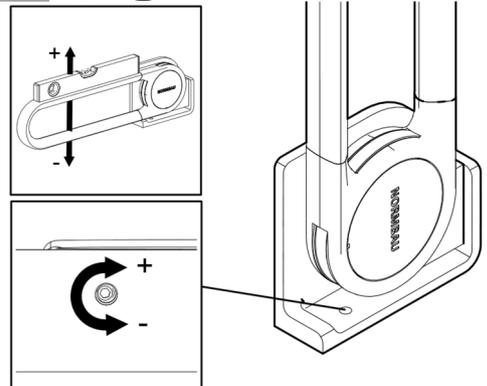
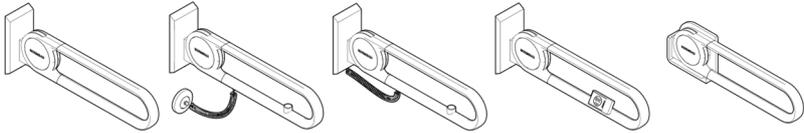
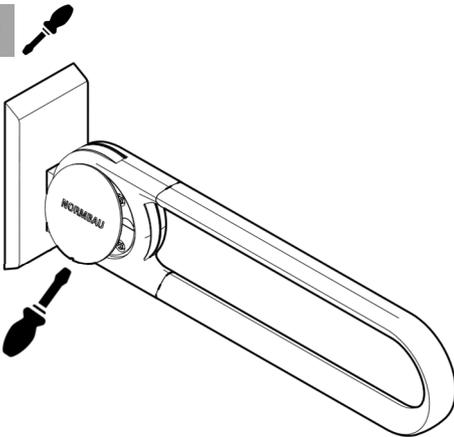
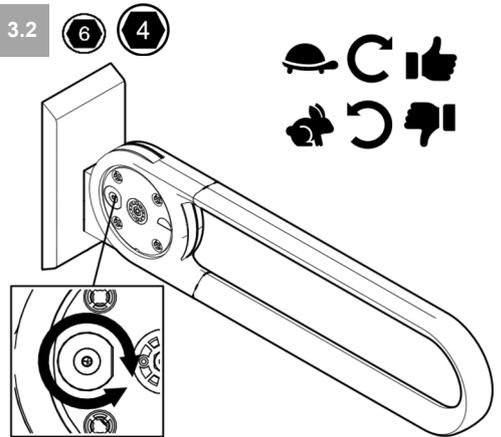
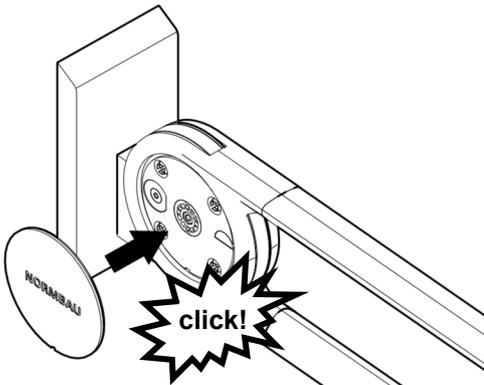
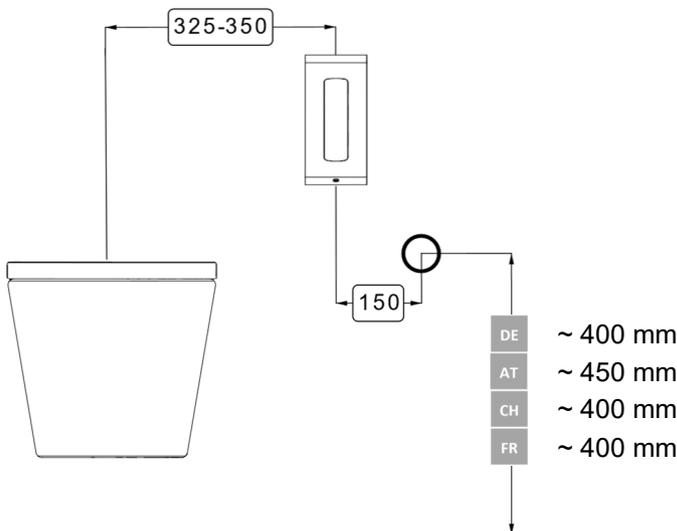
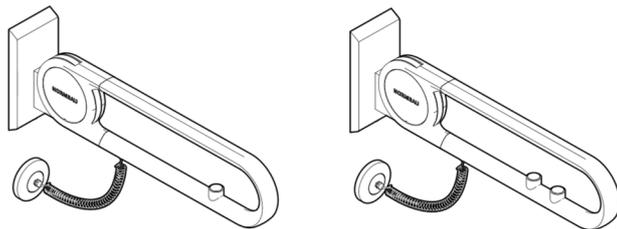
EN

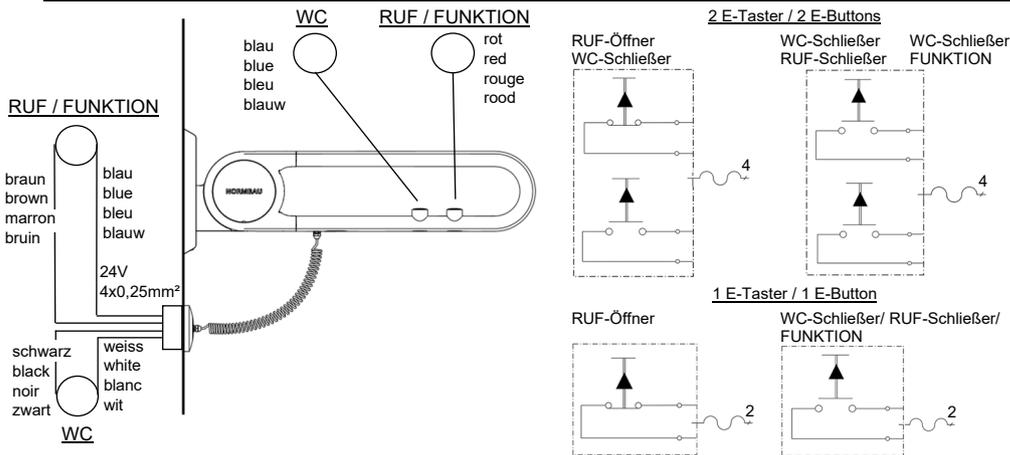
On walls for masonry:
62mm dia. recess for flush-mounted socket
On lightweight walls:
milled hole diameter 68mm dia. for cavity wall socket

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EN

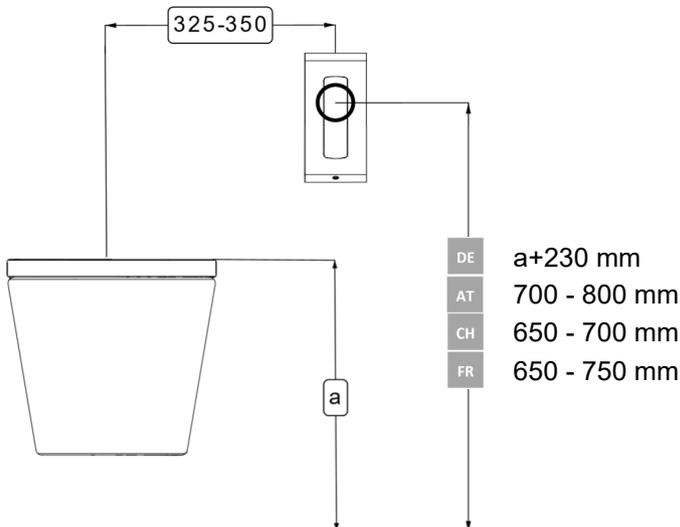
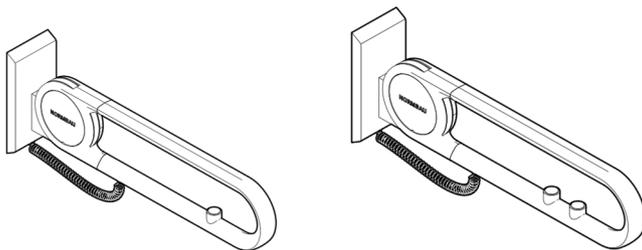
If the e-switch is connected as a call button on an existing intercom according to VDE 0834, a cable relief must be fitted in the junction box. E-Button standard configuration “normally open contact”, also available in configuration “normally closed contact”. A minimum of protection type IP X4 must be provided for the junction box (junction box Up6). The permitted maximum voltage for WC controls or for emergency call systems is 24V.

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On walls for masonry:

EN

62mm dia. recess for flush-mounted socket

On lightweight walls:

milled hole diameter 68mm dia. for cavity wall socket

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NORMBAU GmbH