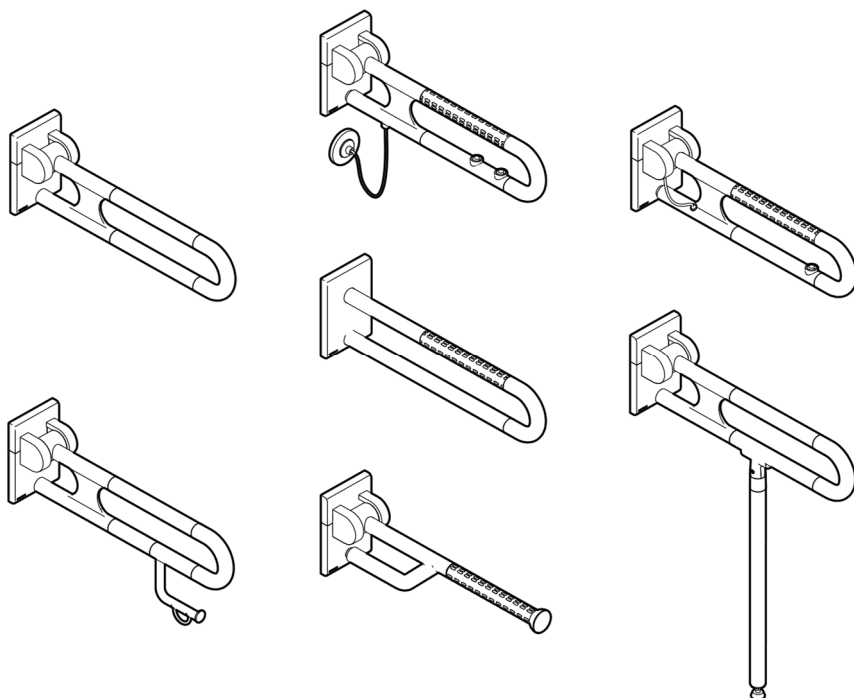
NORMBAU

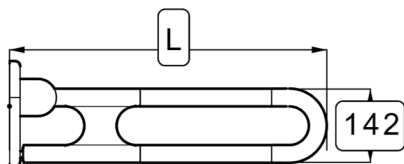
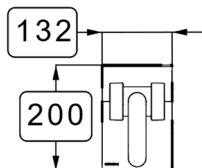
PASSION FOR CARE

Nylon Care 300/400


- DE** Montage- und
Gebrauchsanleitung
- EN** Fixing instruction and
directions for use
- FR** Instructions de montage et
d'utilisation
- NL** Montage- en
gebruiksaanwijzing


Weitere Informationen
Further information
Plus d'informations
Meer informatie
extranet.normbau.de


L (mm)	KG	KG
600	3,6	4
725	4	4,5
850	4,4	4,9
900	4,6	5

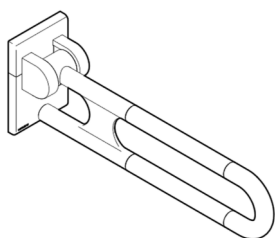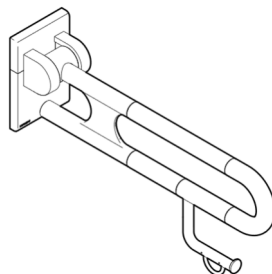
0300 471
0300 472
0300 473
0300 474
0447 010*
0447 030*
0447 050*
0447 070*

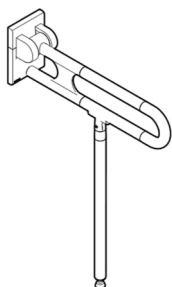
0300 475
0300 476
0300 477
0300 478
0447 120*
0447 140*
0447 160*
0447 180*

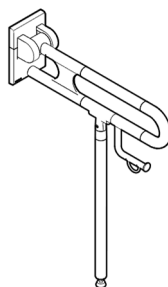
0300 431
0300 432
0300 433
0300 434
0300 451
0300 452
0300 453
0300 454
0447 310*
0447 330*
0447 350*
0447 370*

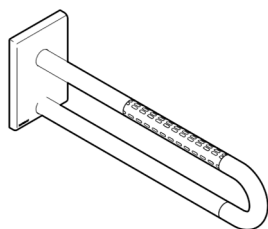
0300 435
0300 436
0300 437
0300 438
0300 455
0300 456
0300 457
0300 458


0467 010
0467 030
0467 050

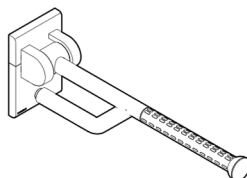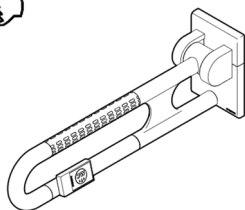
0437 010
0437 030
0437 050
0437 070
0437 110
0437 130
0437 150
0437 170


0447 361
0447 363
0447 365
0447 367

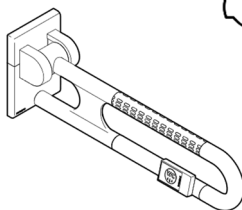
0447 371
0447 373
0447 375
0447 377

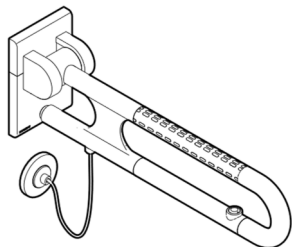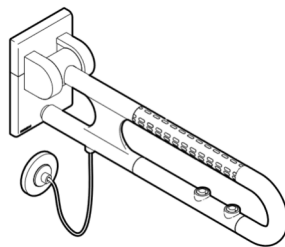
0442 761
0442 763
0442 766
0442 771
0442 773
0442 776
0442 781
0442 783
0442 786
0442 791
0442 793
0442 796

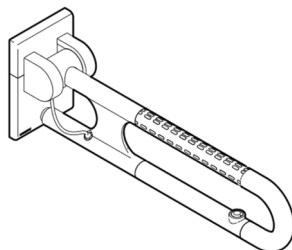
0442 765
0442 767
0442 775
0442 777
0442 785
0442 787
0442 795
0442 797

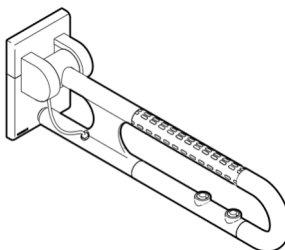
0442 861
0442 863
0442 866
0442 871
0442 873
0442 876
0442 881
0442 883
0442 886
0442 891
0442 893
0442 896

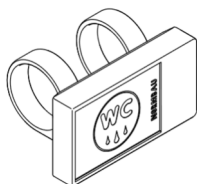0442 865
0442 867
0442 875
0442 877
0442 885
0442 887
0442 895
0442 897

0449 010

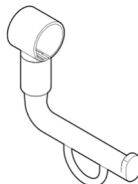0449 120

*

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Intended use

Lift-up support rails and wall support rails are suitable as aids for sanitary equipment in the washing or toilet area. The technical handle systems offer support and holding options and support independent and safe movements for people with physical disabilities.

Disclaimer of liability

NORMBAU assumes no liability for personal injury or property damage that arises or has arisen as a result of improper assembly, use or cleaning, in particular in disregard of the warning and safety instructions listed in these instructions.

Serious incidents that occur in connection with the product must be reported to the manufacturer and the competent authority of the country in which the user is established.

Warranty

The warranty period is 2 years in accordance with current regulations.

If the intended operating conditions and the maintenance and cleaning guidelines are respected, the expected service life is at least 10 years. The product has been tested and validated based on this service life.

Warning notices

There is no toxic hazard from the material used by NORMBAU. Tested according to DIN EN ISO 10993-5:2009.

Risk of injury

The product surfaces can become very hot when exposed to external heat sources, e.g. direct sunlight or contact with hot water.

Risk of injury

If the max. loading permitted is exceeded, it can lead to permanent damage to the product. In this case, the product must be tested before further use!

Risk of injury

When folding down the seat, ensure that there is no part of the body under the product.

Safety instructions

Installation is to be carried out by a specialist and only with NORMBAU mounting sets or suitable fastening material. Before installation, the wall structure must be checked to determine the fastening material. Observe the information on the max. loading permitted and the instructions for mounting! Following installation, the instructions must be made available to the user.

In case of questions, more information, uncertainties or if damage occurs, contact the NORMBAU support service +49 (0) 7843 704 0 or visit us at www.normbau.de.

Load

Max. loading permitted of listed products: 100kg

0300 xxx 0437 xxx 4300 xxx

Max. loading permitted of listed products: 125kg

0442 xxx 0467 xxx 0447 xxx 4442 xxx 4467 xxx 4447 xxx

With max. length and max. loading permitted, the max. force per fastening is 3kN.

Maintenance

NORMBAU products are not maintenance-free! The products are to be tested monthly by qualified specialists for proper function. In the event of defects (e.g. insufficient braking force), the assembly instructions must be observed.

Maintenance and cleaning

Risk of injury

Improper cleaning can cause damage to the surface and lead to injuries.

Nylon products from NORMBAU have a smooth, high-gloss, easy-to-clean surface. Dirty parts can be cleaned with a soft, damp cloth or sponge. If heavily soiled, we recommend a commercially available detergent. Common disinfectants can be used for wipe disinfection. Acidic cleaning agents, care agents, disinfectants and decalcifying agents must always be used in accordance with the instructions for use. An acid content of more than 10% can attack the surface and negatively change the brilliance and color. Use all agents only diluted, leave to act briefly and rinse off sufficiently with water. Abrasive, strongly acidic and alcoholic cleaning and care products should be avoided with Nylon.

More information about
maintenance and cleaning

Disposal

Disposal of the product and packaging is subject to the relevant national regulations and must be disposed of in the prescribed manner.

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GB

Pre-wall installations with panels and integrated mounting plate made of hardwood.

Lightweight walls, made of gypsum, flake board and hardboard with a backing made of at least 30 mm thick multi-layer glued wood.

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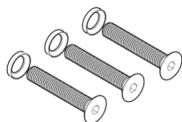
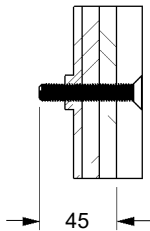
GB

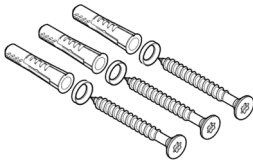
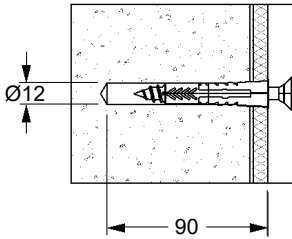
Pre-wall installations with steel mounting plate.

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

GB Masonry made of solid stone:
concrete \geq B15, solid brick \geq Mz 12,
sand-lime brick \geq KS 12,
natural stone with compact grain

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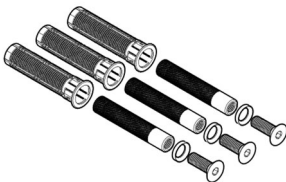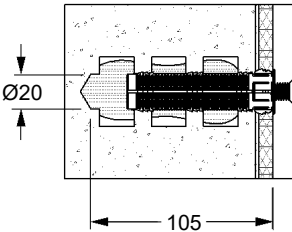
GB Masonry made of perforated stone:
perforated brick HLz \geq 12, cavity brick KSL \geq 12,
checker brick \geq Hbl4, Hbn4, other hollow brick
materials, concrete / lightweight concrete,
solid brick \geq Mz 12, sand-lime brick \geq KS 12,
natural stone with compact grain, solid pumice

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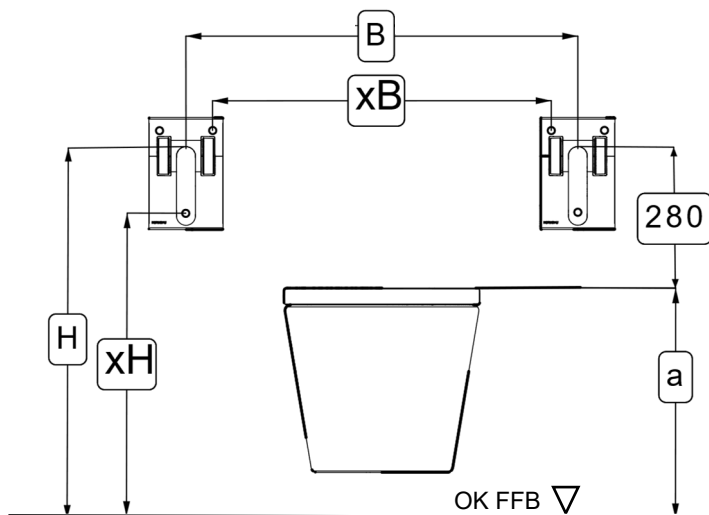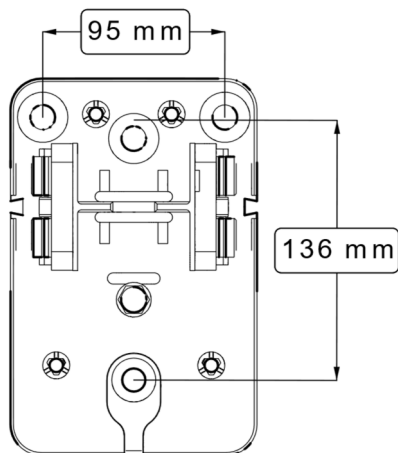
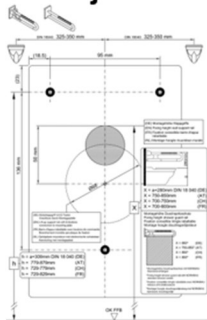
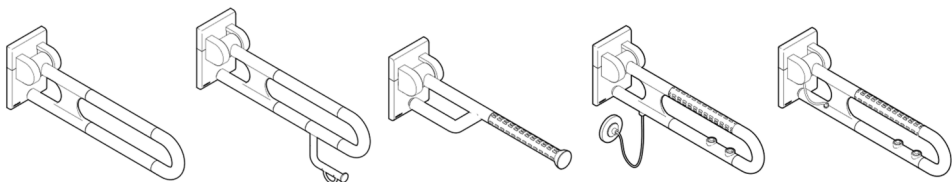
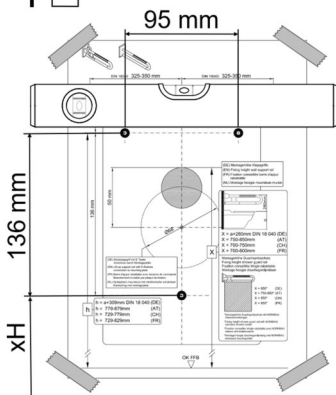
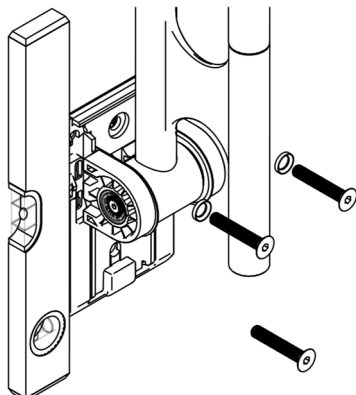
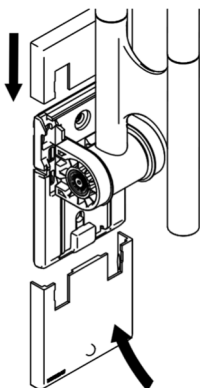
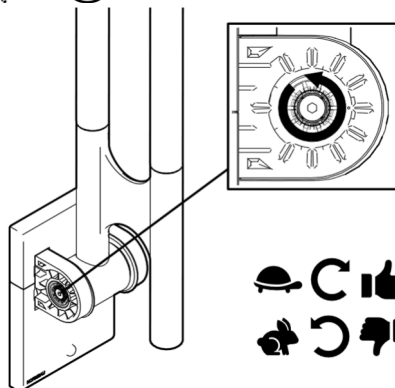
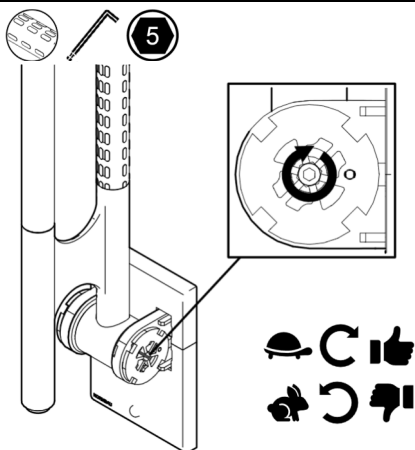
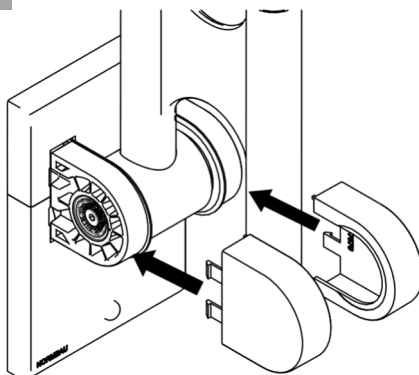
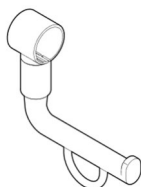
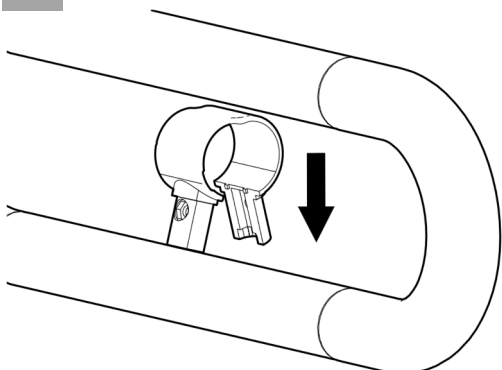
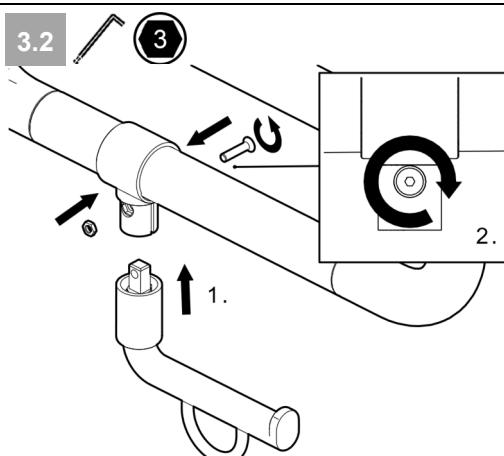
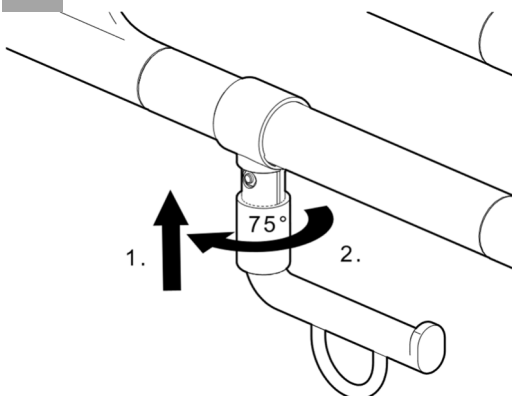
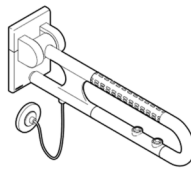
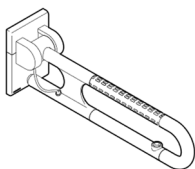
EN ! Risk of injury due to electrocution

Lift-up support rails with E-buttons must be installed outside the exposed water splash zone. Installation may only be carried out by qualified electricians, in compliance with the applicable regulations in accordance with VDE 0100 or the relevant national regulations.

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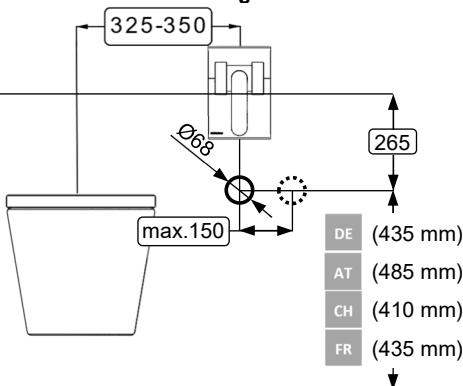
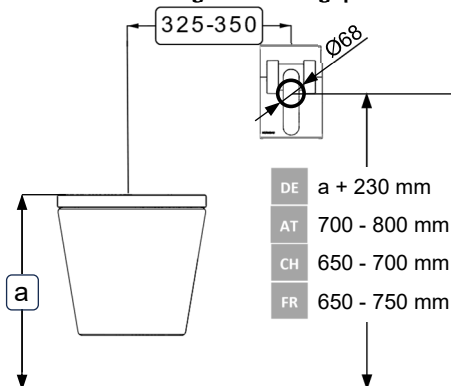
EN The hole diameter for the switch box is Ø68 mm.

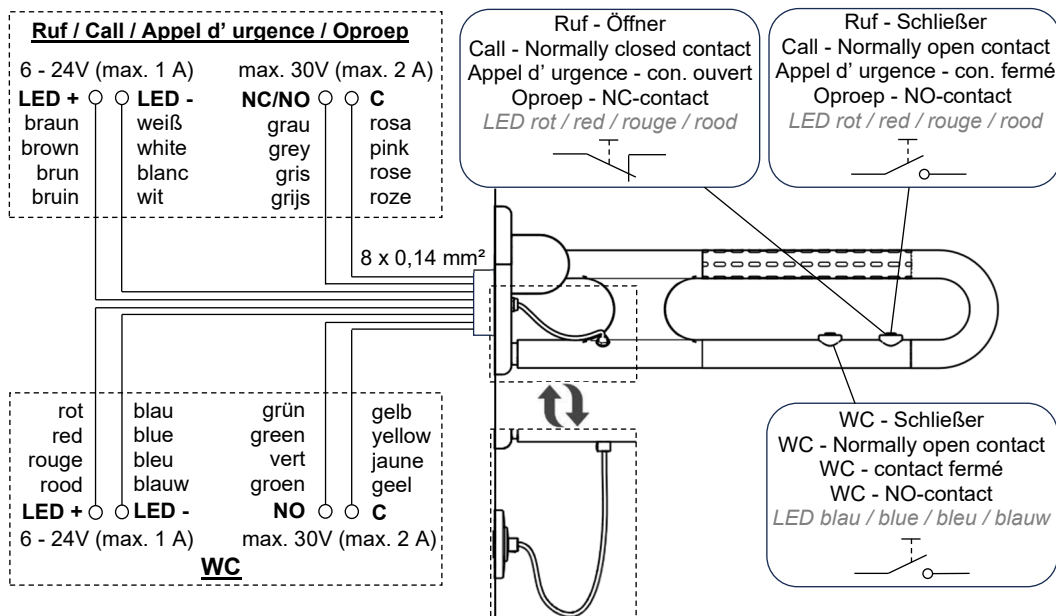
When installing a transformer, a depth of at least 55 mm is recommended.

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A maximum switching capacity of 30 V and 2 A safety extra-low voltage is permitted for WC controls or emergency call systems. If the e-switch is connected as a call button on an existing intercom according to VDE 0834 or the relevant national regulations, a cable relief must be fitted in the junction box. A minimum of protection type IP X4 must be provided for the junction box (junction box UP6).

A direct current voltage supply of between 6V and 24V (max. 1 A) is required to operate the LED lights.

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NORMBAU GmbH