
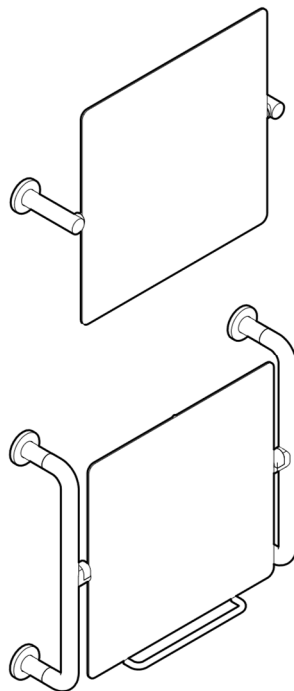
# NORMBAU


PASSION FOR CARE

**Cavere<sup>®</sup> Care**  
**Inox Care<sup>®</sup>**  
**Nylon Care 300/400**


- DE** Montage- und  
Gebrauchsanleitung
- EN** Fixing instruction and  
directions for use
- FR** Instructions de montage et  
d'utilisation
- NL** Montage- en  
gebruiksaanwijzing

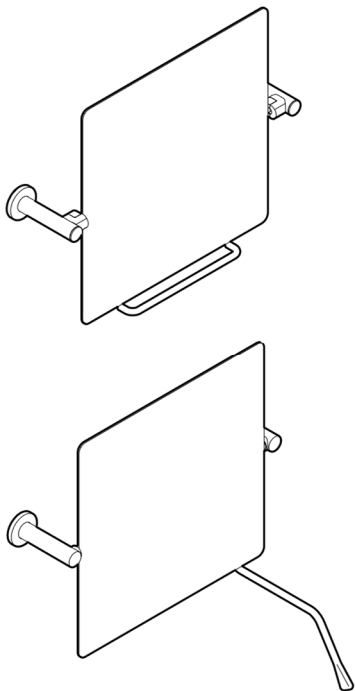
Weitere Informationen  
Further information  
Plus d'informations  
Meer informatie  
[extranet.normbau.de](http://extranet.normbau.de)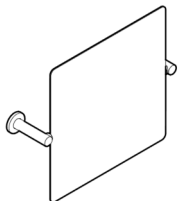


0300 661

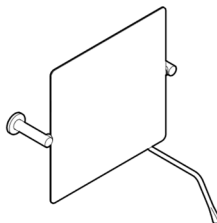
0300 662

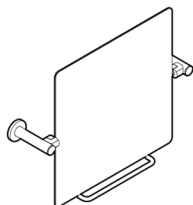
0466 110  
2956 110  
7466 112

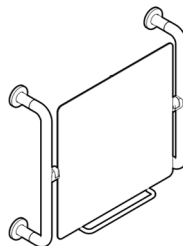0466 010

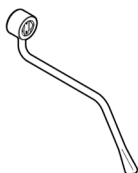0300 665

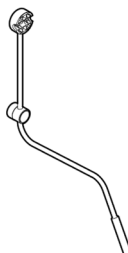0466 450

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### **Intended use**

NORMBAU products are designed for use in wet rooms, at home, in care facilities and hospitals. The tilt function of the adjustable mirrors allows the mirror to be set at an adjustable angle so it can be used by persons standing or sitting.

### **Disclaimer of liability**

NORMBAU assumes no liability for personal injury or property damage that arises or has arisen as a result of improper assembly, use or cleaning, in particular in disregard of the warning and safety instructions listed in these instructions.

### **Warranty**

The statutory warranty period is 2 years.

NORMBAU provides a total of 5 years warranty on functionality and stability.

If the intended operating conditions and the maintenance and cleaning guidelines are respected, the expected service life is at least 10 years. The product has been tested and validated based on this service life.

### **Warning notices**

There is no toxic hazard from the material used by NORMBAU. Tested according to DIN EN ISO 10993-5:2009. Product surfaces can become very hot when being exposed to external heat sources.

### **Safety instructions**

Installation is to be carried out by a specialist and only with NORMBAU mounting sets or suitable fastening material. Before installation, the wall structure must be checked to determine the fastening material. Following installation, the instructions must be made available to the user.

In case of questions, more information, uncertainties or if damage occurs, contact the NORMBAU support service +49 (0) 7843 704 0 or visit us at [www.normbau.de](http://www.normbau.de).

### **Maintenance**

NORMBAU products are maintenance-free.

### **Care and cleaning instructions**

#### **Risk of injury**

Improper cleaning can cause damage to the surface and lead to injuries.

## Cavere Care

Powder-coated products from NORMBAU have a smooth, easy-to-clean surface. In order to maintain the decorative design and color of the coating of our products for years, cleaning and care must be carried out professionally. Slightly adhering dirt can be removed with a sponge and water to which a neutral cleaning agent (e.g. dishwashing liquid) is added. Immediately after each cleaning process, rinse with pure, cold water. A chamois or microfiber cloth removes the remaining wiping marks or drops of water. Abrasive or strong acidic/alkaline cleaning agents should never be used. With metallic coatings, heavy soiling should be cleaned as soon as possible, since dried and old dirt can often only be removed abrasively and with the risk of damage to the powder coating or a change in color. Common wiping disinfectants can be used.

## Inox Care

Stainless steel products from NORMBAU have a smooth, easy-to-clean surface. The material has proven itself both with frequent external loads and when used in a corrosive environment. However, it should be noted that stainless steel can also rust without care and by the action of aggressive agents. Damage to the passive protective layer created by oxygen must be avoided. Dirty parts can be cleaned with a soft, damp cloth or sponge. If heavily soiled, we recommend a commercially available detergent. Common wiping disinfectants can be used. Cleaners with hydrochloric acid, citric acid and acetic acid (tile cleaner) and cleaning sponges made of steel wool are not suitable for use on stainless steel.

## Nylon Care 300/400

Nylon products from NORMBAU have a smooth, high-gloss, easy-to-clean surface. Dirty parts can be cleaned with a soft, damp cloth or sponge. If heavily soiled, we recommend a commercially available detergent. Common disinfectants can be used for wipe disinfection. Acidic cleaning agents, care agents, disinfectants and decalcifying agents must always be used in accordance with the instructions for use. An acid content of more than 10% can attack the surface and negatively change the brilliance and color. Use all agents only diluted, leave to act briefly and rinse off sufficiently with water. Abrasive, strongly acidic and alcoholic cleaning and care products should be avoided with Nylon.

More information about  
maintenance and cleaning

## Disposal

Disposal of the product and packaging is subject to the relevant national regulations and must be disposed of in the prescribed manner.

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**NORMBAU GmbH**