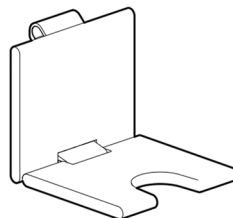
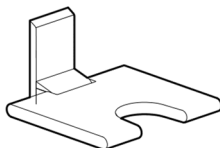
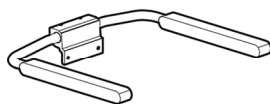
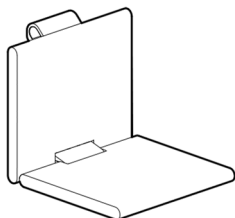
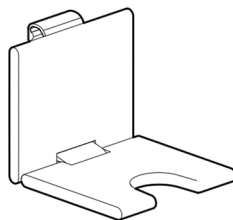
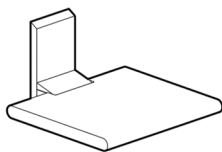
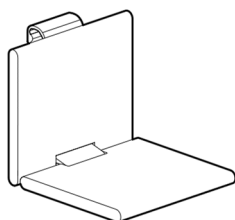
NORMBAU

PASSION FOR CARE

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Intended use

NORMBAU products are designed for use in wet rooms, at home, in care facilities and hospitals. They offer support to people with and without physical limitations during showering.

Disclaimer of liability

NORMBAU assumes no liability for personal injury or property damage that arises or has arisen as a result of improper assembly, use or cleaning, in particular in disregard of the warning and safety instructions listed in these instructions. According to the current regulation, the warranty is two years.

Under the intended conditions of use and in compliance with the guidelines for maintenance and cleaning, the intended lifespan of the article is five years.

Warning notices

There is no toxic hazard from the material used by NORMBAU. Tested according to DIN EN ISO 10993-5:2009. Product surfaces can become very hot when exposed to external heat sources.

Risk of injury

If the max permissible user weight is exceeded, it can lead to permanent damage to the product. In this case, the product must be tested before further use!

Risk of injury

When folding the lift-up shower seat, do not put your hand between the wall and the shower seat!

Safety instructions

Installation is to be carried out by a specialist and only with NORMBAU mounting sets or suitable fastening material. Before installation, the wall structure must be checked to determine the fastening material. Observe the information on the max. loading permitted and the instructions for mounting! Following installation, the instructions must be made available to the user.

In case of questions, more information, uncertainties or if damage occurs, contact the NORMBAU support service +49 (0) 7843 704 0 or visit us at www.normbau.de.

Load

Max. loading permitted of listed products: 150kg

Maintenance

NORMBAU products are not maintenance-free! The products are to be tested monthly by qualified specialists for proper function. In the event of defects (e.g. insufficient braking force), the assembly instructions must be observed.

Care and cleaning instructions

Risk of injury

Improper cleaning can cause damage to the surface and lead to injuries.

Cavere Care

Powder-coated products from NORMBAU have a smooth, easy-to-clean surface. In order to maintain the decorative design and color of the coating of our products for years, cleaning and care must be carried out professionally. Slightly adhering dirt can be removed with a sponge and water to which a neutral cleaning agent (e.g. dishwashing liquid) is added. Immediately after each cleaning process, rinse with pure, cold water. A chamois or microfiber cloth removes the remaining wiping marks or drops of water. Abrasive or strong acidic/alkaline cleaning agents should never be used. With metallic coatings, heavy soiling should be cleaned as soon as possible, since dried and old dirt can often only be removed abrasively and with the risk of damage to the powder coating or a change in color. Common wiping disinfectants can be used.

Cavere Chrome Care

Abrasive sponges and scouring agents should not be used to clean our Cavere Chrome products. We also advise against the use of cleaners containing solvent or acid, limescale removers, vinegar and cleaning agents containing acetic acid. The chrome surface can be cleaned using a little soap on a damp cloth. Limescale stains can be avoided by drying the tap after every use. If limescale deposits still build up, you can remove these using an environmentally friendly cleaning agent. The appropriate cleaning agent should only be left on for a short time before being rinsed thoroughly with water. In areas with hard water, it may be necessary to clean the chrome surface at specific intervals. Please observe the following points when cleaning:

- Residues from toiletries such as shampoo, shower gel or liquid soap should be removed quickly as these can leave permanent stains.
- Harsh cleaning methods and aggressive cleaning agents can damage surfaces and even penetrate the inside of the products.
- Do not apply spray cleaners directly to the chrome surface. Spray the cleaning agent onto a soft cloth and use this to wipe the surface instead.
- Use clean water to rinse the chrome surface thoroughly after cleaning.

- Do not use any cleaning agents that contain hydrochloric acids, formic acids, chlorine bleaching or acetic acids, as these can cause considerable damage.
- Be sure to treat limescale stains and other types of soiling correctly. Any damage caused as a result of improper treatment shall not be covered under warranty.

PUR-integral foam

The upholstered seats should ideally be treated with mild, over-the-counter cleaning agents and disinfectants with low alcohol content. Agents containing iodine can lead to staining, and long exposure times for coloured care products should always be avoided. Avoid exposing upholstered seats to temperatures over 80°C.

Disposal

Disposal of the product and packaging is subject to the relevant national regulations and must be disposed of in the prescribed manner.

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NORMBAU GmbH Schwarzwaldstr. 15 77871 Renchen DE
NORMBAU GmbH