
NORMBAU

PASSION FOR CARE

Special Care
PSY

- DE Montage- und
Gebrauchsanleitung
- EN Fixing instruction and
directions for use
- FR Instructions de montage et
d'utilisation
- NL Montage- en
gebruiksaanwijzing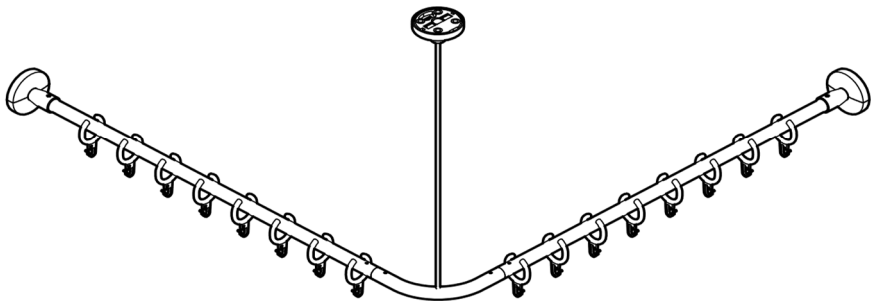

Weitere Informationen
Further information
Plus d'informations
Meer informatie
extranet.normbau.de


0788 001
 0788 002
 2088 001
 2088 002

0788 005
 0788 006
 0788 007
 2088 005
 2088 006
 2088 007

0788 010
 0788 020
 0788 030
 0788 040
 0788 050
 0788 060
 0788 070
 0788 080
 2088 010
 2088 020
 2088 030
 2088 040
 2088 050
 2088 060
 2088 070
 2088 080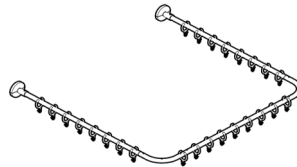

0788 210
 0788 220
 0788 230
 0788 240
 0788 250
 0788 260
 0788 270
 0788 280
 2088 210
 2088 220
 2088 230
 2088 240
 2088 250
 2088 260
 2088 270
 2088 280

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Intended use

It is a product to prevent or reduce the risk of suicide.

The individual wall and ceiling attachments disengage directly at the attachment point before the maximum load of 30kg is reached. The disengagement function depends on the environmental and usage conditions and on professional assembly according to the instructions.

During the development of this product various possible (suicide) situations were taking into consideration. Naturally these are numerous and cannot be predicted or simulated in their entirety. The products are therefore only conditionally guaranteed to be "suicide-inhibiting".

Disclaimer of liability

NORMBAU assumes no liability for personal injury or property damage that arises or has arisen as a result of improper assembly, use or cleaning, in particular in disregard of the warning and safety instructions listed in these instructions. According to the current regulation, the warranty is two years.

Under the intended conditions of use and in compliance with the guidelines for maintenance and cleaning, the intended lifespan of the article is five years.

Warning notices


There is no toxic hazard from the material used by NORMBAU. Tested according to DIN EN ISO 10993-5:2009. Product surfaces can become very hot when exposed to external heat sources.

Attention!

Fixing has to be done by skilled workers only. If fixing has not been completed professionally or in cases of inappropriate use, NORMBAU will not assume liability for any consequential damages.

Failure to comply with the stated fixing measurements can impair the function of this product and lead to severe injuries or death! After fixing this product, the installation company, the disengagement function of max. 30kg on the wall rosettes and on the ceiling suspensions to be tested individually. Oblique or wrong fixing of the wall mounting can impair the function of this product and lead to severe injuries or death!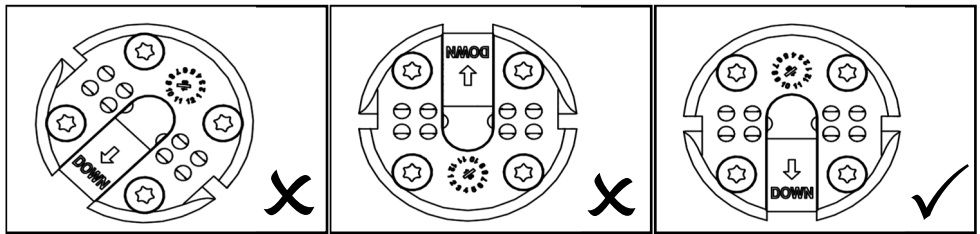

Installation is to be carried out by a specialist and only with NORMBAU fixing sets or suitable fastening material. Before installation, the wall structure must be checked to determine the fastening material.

⚠ Risk of injury

Due to the anti-ligature characteristic, the shower curtain rail can come down from the ceiling when loaded.

Safety instructions

Observe the information on the max. loading permitted and the instructions for mounting!

Following installation, the instructions must be made available to the user.

In case of questions, more information, uncertainties or if damage occurs, contact the NORMBAU support service +49 (0) 7843 704 0 or visit us at www.normbau.de.

Maintenance

NORMBAU products are maintenance-free.

Care and cleaning instructions

⚠ Risk of injury

Improper cleaning can cause damage to the surface and lead to injuries.

Nylon products from NORMBAU have a smooth, high-gloss, easy-to-clean surface. Dirty parts can be cleaned with a soft, damp cloth or sponge. If heavily soiled, we recommend a commercially available detergent. Common disinfectants can be used for wipe disinfection. Acidic cleaning agents, care agents, disinfectants and decalcifying agents must always be used in accordance with the instructions for use. An acid content of more than 10% can attack the surface and negatively change the brilliance and color. Use all agents only diluted, leave to act briefly and rinse off sufficiently with water. Abrasive, strongly acidic and alcoholic cleaning and care products should be avoided with Nylon.

Powder-coated products from NORMBAU have a smooth, easy-to-clean surface. In order to maintain the decorative design and color of the coating of our products for years, cleaning and care must be carried out professionally. Slightly adhering dirt

can be removed with a sponge and water to which a neutral cleaning agent (e.g. dishwashing liquid) is added. Immediately after each cleaning process, rinse with pure, cold water. A chamois or microfiber cloth removes the remaining wiping marks or drops of water. Abrasive or strong acidic/alkaline cleaning agents should never be used. With metallic coatings, heavy soiling should be cleaned as soon as possible, since dried and old dirt can often only be removed abrasively and with the risk of damage to the powder coating or a change in color. Common wiping disinfectants can be used.

Stainless steel products from NORMBAU have a smooth, satin finish, easy-to-clean surface. The material has proven itself both with frequent external loads and when used in a corrosive environment. However, it should be noted that stainless steel can also rust without care and by the action of aggressive agents. Damage to the passive protective layer created by oxygen must be avoided. Dirty parts can be cleaned with a soft, damp cloth or sponge. If heavily soiled, we recommend a commercially available detergent. Common wiping disinfectants can be used. Cleaners with hydrochloric acid, citric acid and acetic acid (tile cleaner) and cleaning sponges made of steel wool are unsuitable for use on stainless steel.

Disposal

Disposal of the product and packaging is subject to the relevant national regulations and must be disposed of in the prescribed manner.

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NORMBAU GmbH