
NORMBAU


PASSION FOR CARE

Cavere[®] Care


- DE** Montage- und
Gebrauchsanleitung
EN Fixing instruction and
directions for use
FR Instructions de montage et
d'utilisation
NL Montage- en
gebruiksaanwijzing

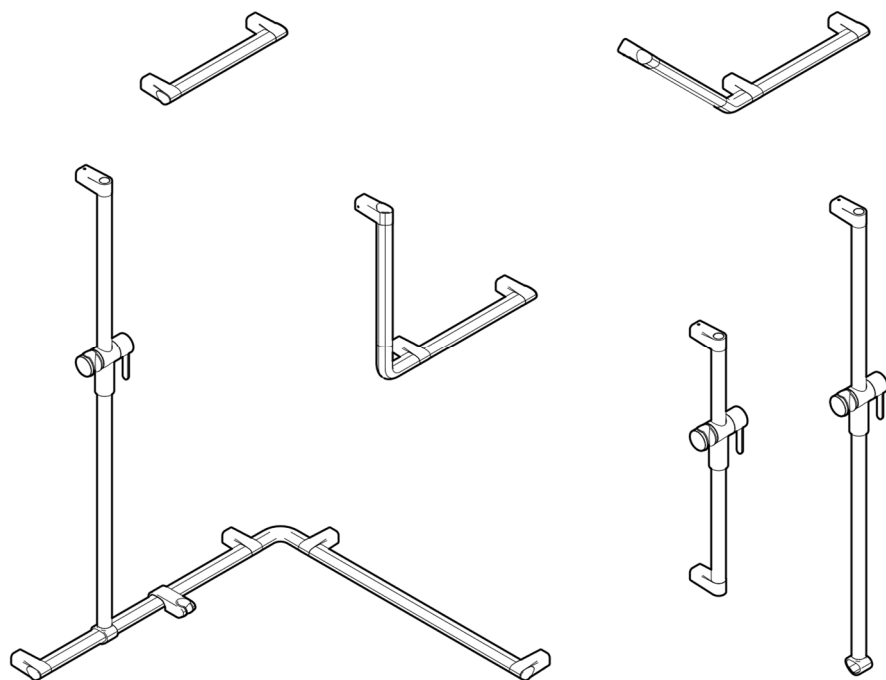
Weitere Informationen
Further information
Plus d'informations
Meer informatie
extranet.normbau.de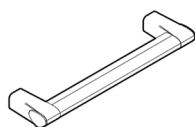


7453 030
7453 040
7453 060
7453 080
7453 100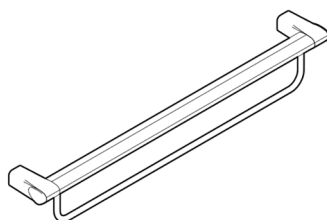


7453 160
7453 180

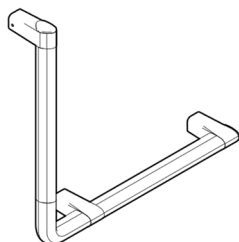
7464 001
7464 002
7464 011
7464 012
7464 021
7464 022
7464 031
7464 032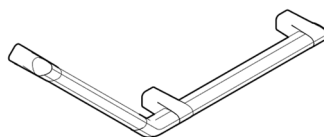


7464 130
7464 131
7464 160
7464 161

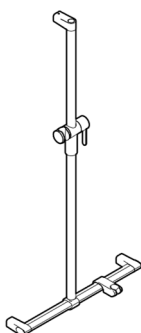
7485 053
7485 054
7485 061
7485 091

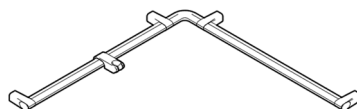
7484 054
7484 055
7484 056
7484 076

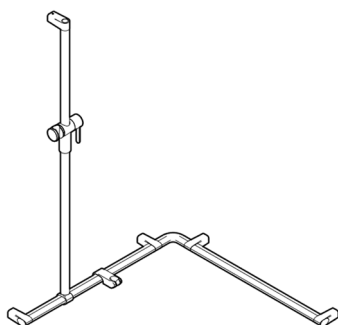
7486 064
7486 065
7486 066
7486 067
7486 115
7486 116
7486 117
7486 118
7486 119
7486 120
7486 121
7486 122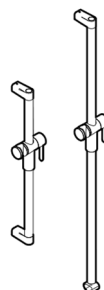

7488 060
7488 119
7488 110
7488 120

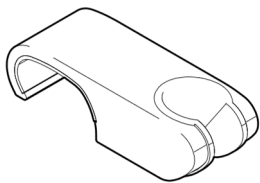7485 030

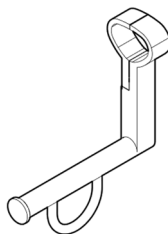7449 110

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Intended use

Grab bars and grab rails as well as shower handrail systems are suitable as aids in the bathroom area. The technical handle systems offer support and holding options and support independent and safe movements for people with physical disabilities.

Disclaimer of liability

NORMBAU assumes no liability for personal injury or property damage that arises or has arisen as a result of improper assembly, use or cleaning, in particular in disregard of the warning and safety instructions listed in these instructions.

Serious incidents that occur in connection with the product must be reported to the manufacturer and the competent authority of the country in which the user is established.

Warranty

The statutory warranty period is 2 years.

NORMBAU provides a total of 5 years warranty on functionality and stability.

If the intended operating conditions and the maintenance and cleaning guidelines are respected, the expected service life is at least 10 years. The product has been tested and validated based on this service life.

Warning notices

There is no toxic hazard from the material used by NORMBAU. Tested according to DIN EN ISO 10993-5:2009.

Risk of injury

The product surfaces can become very hot when exposed to external heat sources, e.g. direct sunlight or contact with hot water.

Risk of injury

If the max. loading permitted is exceeded, it can lead to permanent damage to the product. In this case, the product must be tested before further use!

Safety instructions

Installation is to be carried out by a specialist and only with NORMBAU fixing sets or suitable fastening material. Before installation, the wall structure must be checked to determine the fastening material. Observe the information on the max. loading permitted and the instructions for mounting! Following installation, the instructions must be made available to the user.

In case of questions, more information, uncertainties or if damage occurs, contact the NORMBAU support service +49 (0) 7843 704 0 or visit us at www.normbau.de.

Load

Max. loading permitted for the products listed below: 150 kg

7484 054	7485 053	7486 064	7486 115	7486 119
7484 055	7485 054	7486 065	7486 116	7486 120
7484 056	7485 061	7486 066	7486 117	7486 121
7484 076	7485 091	7486 067	7486 118	7486 122

Max. loading permitted for the products listed below: 100 kg

7453 030	7453 160	7464 012	7464 130	7488 060
7453 040	7453 180	7464 021	7464 131	7488 119
7453 060	7464 001	7464 022	7464 160	7488 110
7453 080	7464 002	7464 031	7464 161	7488 120
7453 100	7464 011	7464 032		

Maintenance

NORMBAU products are maintenance-free.

Maintenance and cleaning

Risk of injury

Improper cleaning can cause damage to the surface and lead to injuries.

For hygiene reasons and to ensure proper cleaning in accordance with the instructions, hanging seats and the horizontal shower head holder must be removed from the shower handrail regularly to maintain a healthy surface structure and a durable coating.

Cavere Care

Powder-coated products from NORMBAU have a smooth, easy-to-clean surface. In order to maintain the decorative design and color of the coating of our products for years, cleaning and care must be carried out professionally. Slightly adhering dirt can be removed with a sponge and water to which a neutral cleaning agent (e.g. dishwashing liquid) is added. Immediately after each cleaning process, rinse with pure, cold water. A chamois or microfiber cloth removes the remaining wiping marks or drops of water. Abrasive or strong acidic/alkaline cleaning agents should never be used. With metallic coatings, heavy soiling should be cleaned as soon as possible, since dried and old dirt can often only be removed abrasively and with the risk of damage to the powder coating or a change in color. Common wiping disinfectants can be used.

More information about
maintenance and cleaning

Disposal

Disposal of the product and packaging is subject to the relevant national regulations and must be disposed of in the prescribed manner.

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