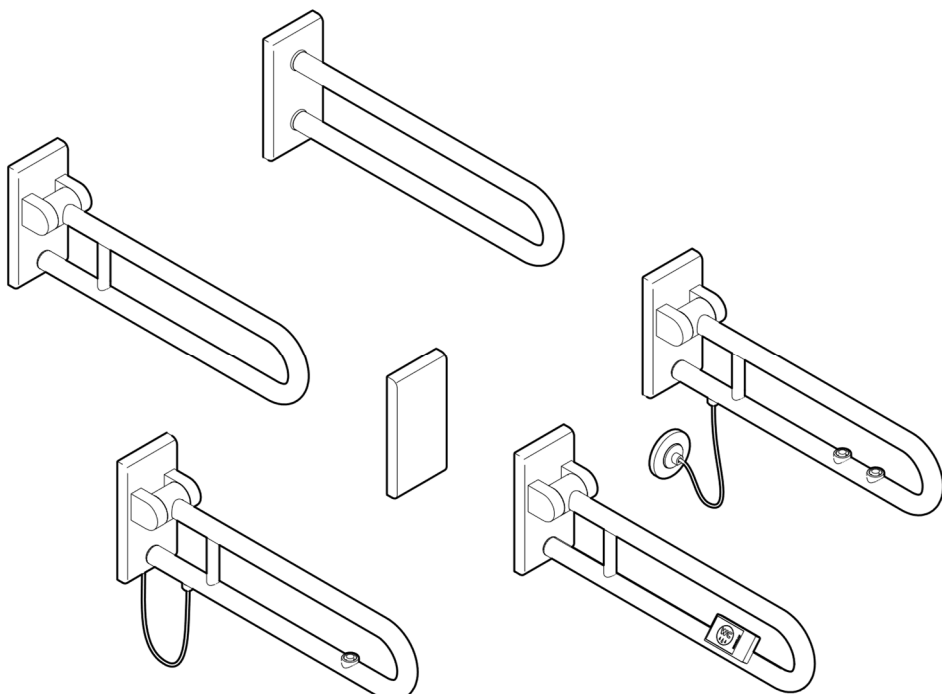
# NORMBAU

PASSION FOR CARE

Inox Care®

- DE Montage- und  
Gebrauchsanleitung
- EN Fixing instruction and  
directions for use
- FR Instructions de montage et  
d'utilisation
- NL Montage- en  
gebruiksaanwijzing

Weitere Informationen  
Further information  
Plus d'informations  
Meer informatie  
[extranet.normbau.de](http://extranet.normbau.de)

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### **Intended use**

Lift-up support rails and wall support rails are suitable as aids for sanitary equipment in the washing or toilet area. The technical handle systems offer support and holding options and support independent and safe movements for people with physical disabilities.

### **Disclaimer of liability**

NORMBAU assumes no liability for personal injury or property damage that arises or has arisen as a result of improper assembly, use or cleaning, in particular in disregard of the warning and safety instructions listed in these instructions.

Serious incidents that occur in connection with the product must be reported to the manufacturer and the competent authority of the country in which the user is established.

### **Warranty**

The warranty period is 2 years in accordance with current regulations.

If the intended operating conditions and the maintenance and cleaning guidelines are respected, the expected service life is at least 10 years. The product has been tested and validated based on this service life.

### **Warning notices**

There is no toxic hazard from the material used by NORMBAU. Tested according to DIN EN ISO 10993-5:2009.

#### **Risk of injury**

The product surfaces can become very hot when exposed to external heat sources, e.g. direct sunlight or contact with hot water.

#### **Risk of injury**

If the max. loading permitted is exceeded, it can lead to permanent damage to the product. In this case, the product must be tested before further use!

#### **Risk of injury**

When folding down the seat, ensure that there is no part of the body under the product.

### **Safety instructions**

Installation is to be carried out by a specialist and only with NORMBAU fixing sets or suitable fastening material. Before installation, the wall structure must be checked to determine the fastening material. Observe the information on the max. loading permitted and the instructions for mounting! Following installation, the instructions must be made available to the user.

In case of questions, more information, uncertainties or if damage occurs, contact the NORMBAU support service +49 (0) 7843 704 0 or visit us at [www.normbau.de](http://www.normbau.de).

### **Load**

Max. loading permitted of listed products: 125kg

### **Maintenance and cleaning**

NORMBAU products are not maintenance-free! The products are to be tested monthly by qualified specialists for proper function. In the event of defects (e.g. insufficient braking force), the assembly instructions must be observed.

### **Care and cleaning instructions**

#### **Risk of injury**

Improper cleaning can cause damage to the surface and lead to injuries.

Stainless steel products from NORMBAU have a smooth, easy-to-clean surface. The material has proven itself both with frequent external loads and when used in a corrosive environment. However, it should be noted that stainless steel can also rust without care and by the action of aggressive agents. Damage to the passive protective layer created by oxygen must be avoided. Dirty parts can be cleaned with a soft, damp cloth or sponge. If heavily soiled, we recommend a commercially available detergent. Common wiping disinfectants can be used. Cleaners with hydrochloric acid, citric acid and acetic acid (tile cleaner) and cleaning sponges made of steel wool are not suitable for use on stainless steel.

More information about  
maintenance and cleaning

### **Disposal**

Disposal of the product and packaging is subject to the relevant national regulations and must be disposed of in the prescribed manner.

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
**GB** Pre-wall installations, lightweight walls with a backing made of at least 30 mm thick multi-layer glued wood. Pre-drill according to the screw.

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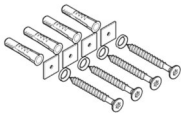
**FR** Maçonnerie en pierre pleine

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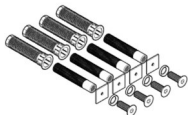
**7470 040 / 050**

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**GB** Masonry made of perforated stone

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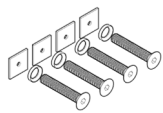**GB** Pre-wall installations with steel mounting plate or metal stud wall, on lightweight walls made of plasterboard, chipboard, hardboard

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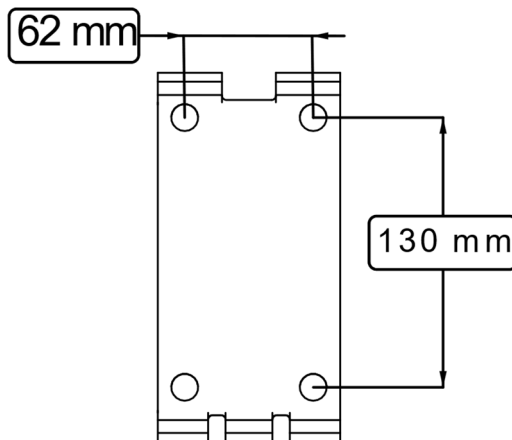
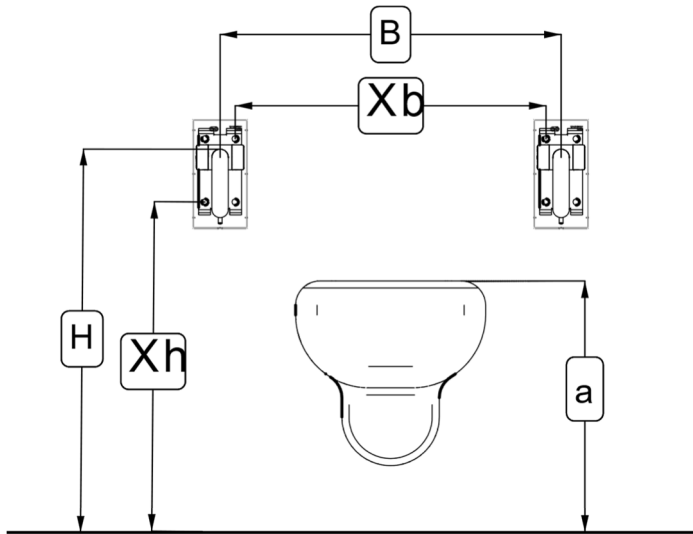
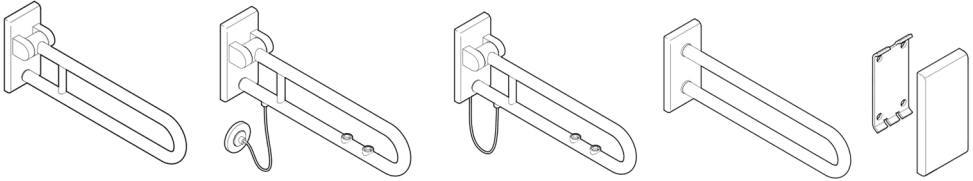
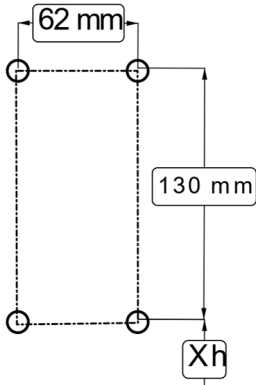
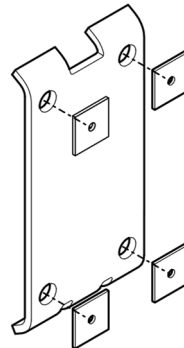
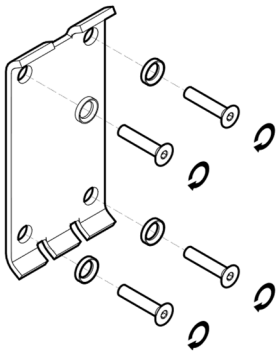
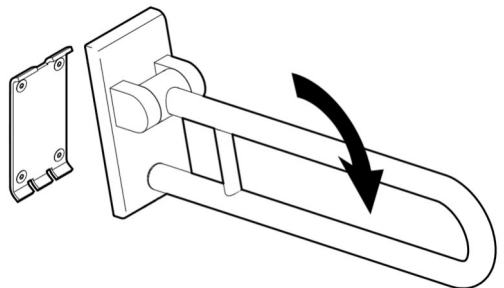
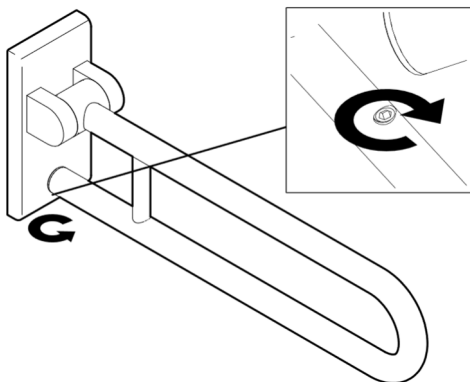
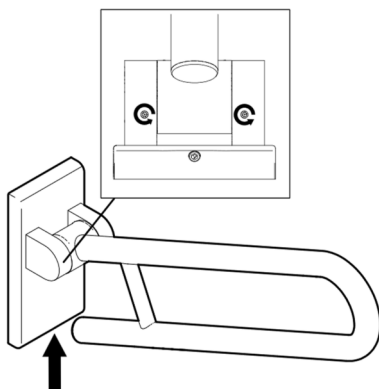
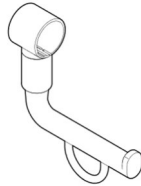
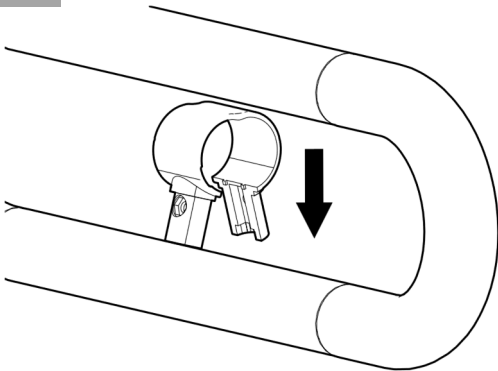
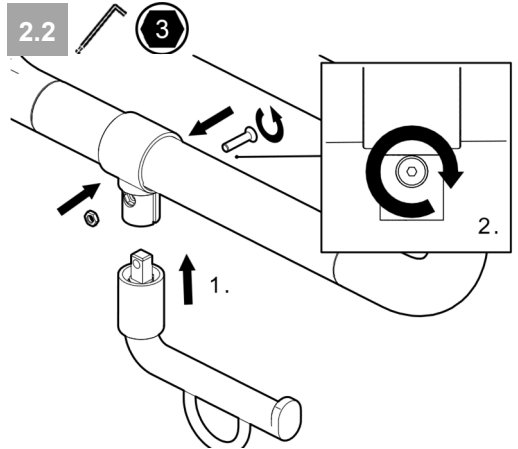
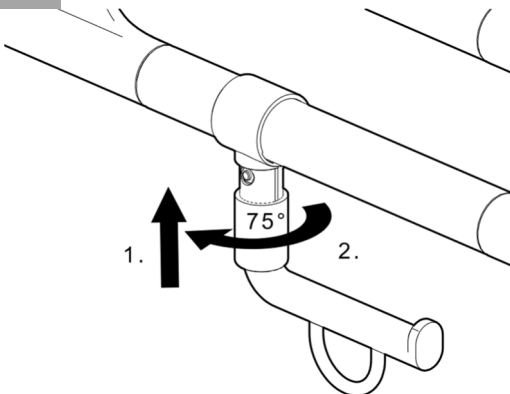
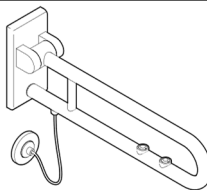
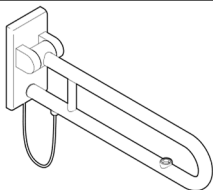
**EN** ⚠ **Risk of injury due to electrocution**

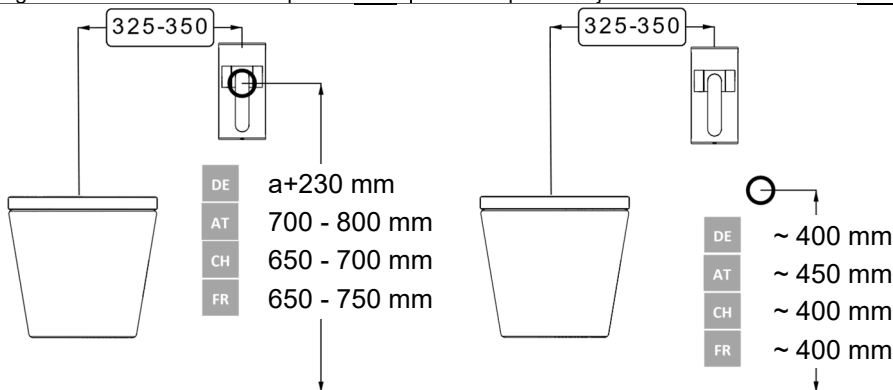
Installation may only be carried out by qualified electricians, in compliance with the applicable regulations in accordance with VDE 0100 or the relevant national regulations.

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On walls for masonry:

**EN**

62mm dia. recess for flush-mounted socket

On lightweight walls:

milled hole diameter 68mm dia. for cavity wall socket

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EN

The permitted maximum voltage for WC controls or for emergency call systems is 24V. If the e-switch is connected as a call button on an existing intercom according to VDE 0834, a cable relief must be fitted in the junction box. A minimum of protection type IP X4 must be provided for the junction box (junction box Up6).

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**NORMBAU GmbH**