
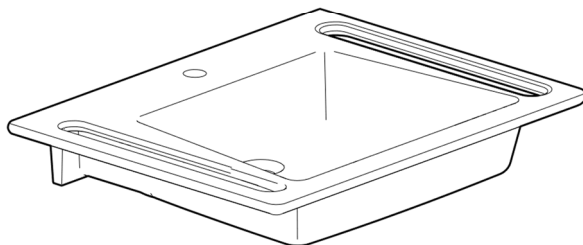
# NORMBAU


PASSION FOR CARE

Care  
Cavere® Care


- DE** Montage- und  
Gebrauchsanleitung
- EN** Fixing instruction and  
directions for use
- FR** Instructions de montage et  
d'utilisation
- NL** Montage- en  
gebruiksaanwijzing

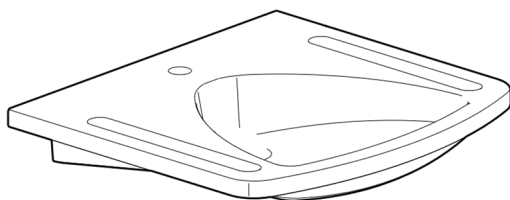
Weitere Informationen  
Further information  
Plus d'informations  
Meer informatie  
[extranet.normbau.de](http://extranet.normbau.de)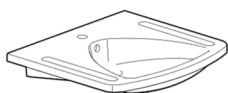


7530 650

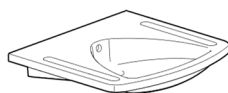
7530 660

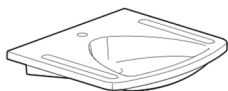
7530 670

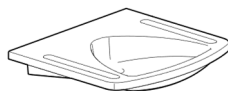
7530 680

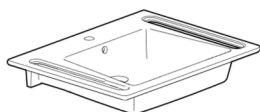
3501 505

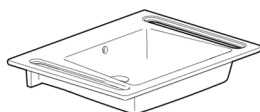3501 506

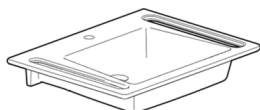3501 500

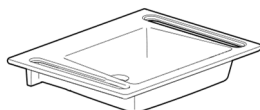3501 501

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### **Intended use**

NORMBAU products are designed for use in wet rooms, at home, in care facilities and hospitals. They provide grip and support and assist the independent standing and sitting of persons with physical impairment.

### **Disclaimer of liability**

NORMBAU assumes no liability for personal injury or property damage that arises or has arisen as a result of improper assembly, use or cleaning, in particular in disregard of the warning and safety instructions listed in these instructions.

### **Warranty**

The warranty period is 2 years in accordance with current regulations.

If the intended operating conditions and the maintenance and cleaning guidelines are respected, the expected service life is at least 10 years. The product has been tested and validated based on this service life.

### **Warning notices**

#### **Risk of injury**

If the max. loading permitted is exceeded, it can lead to permanent damage to the product. In this case, the product must be tested before further use!

There is no toxic hazard from the material used by NORMBAU. Tested according to DIN EN ISO 10993-5:2009. Product surfaces can become very hot when exposed to external heat sources.

### **Safety instructions**

Installation is to be carried out by a specialist and only with NORMBAU fixing sets or suitable fastening material. Before installation, the wall structure must be checked to determine the fastening material. Observe the information on the max. loading permitted and the instructions for mounting! Following installation, the instructions must be made available to the user.

In case of questions, more information, uncertainties or if damage occurs, contact the NORMBAU support service +49 (0) 7843 704 0 or visit us at [www.normbau.de](http://www.normbau.de).

### **Load**

Max. loading permitted of listed products: 150kg

### **Maintenance**

NORMBAU products are maintenance-free.

### **Care and cleaning instructions**

NORMBAU washstands can cope with all the common bathroom cleaning products and cosmetics. Staining, for example with shoe polish, ink, lipstick, nail polish or make-up, can generally be removed. Please note that hair dye residues must be removed immediately. After removal of the protective film, the washstand should be treated with care for 4 weeks until it has fully hardened.

For the day-to-day cleaning of your NORMBAU washstand, you will normally only require warm water and a damp cloth. In order to protect the surface, the water temperature should not exceed 65°C, commercially available mild detergents that do not contain abrasives are recommended. Limescale can be removed using non-abrasive, limescale-dissolving mild detergents. Please only use soft cloths, not microfibre cloths.

Never use cleaning agents containing acid, or that are aggressive or abrasive, as this could damage the surface finish. Residues of strong substances containing fluorine or alcoholic substances must be removed immediately. The inappropriate use of abrasive cleaning agents, contact with sharp, scarifying or abrading objects and the long-term effect of aggressive dyes or heat can damage the surface.

In order to preserve a high quality surface finish, your NORMBAU washstand should be polished once or twice a year using a fine polishing product.

### **⚠ Risk of injury**

Improper cleaning can cause damage to the surface and lead to injuries.

More information about  
maintenance and cleaning

### **Disposal**

Disposal of the product and packaging is subject to the relevant national regulations and must be disposed of in the prescribed manner.

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**NORMBAU GmbH**