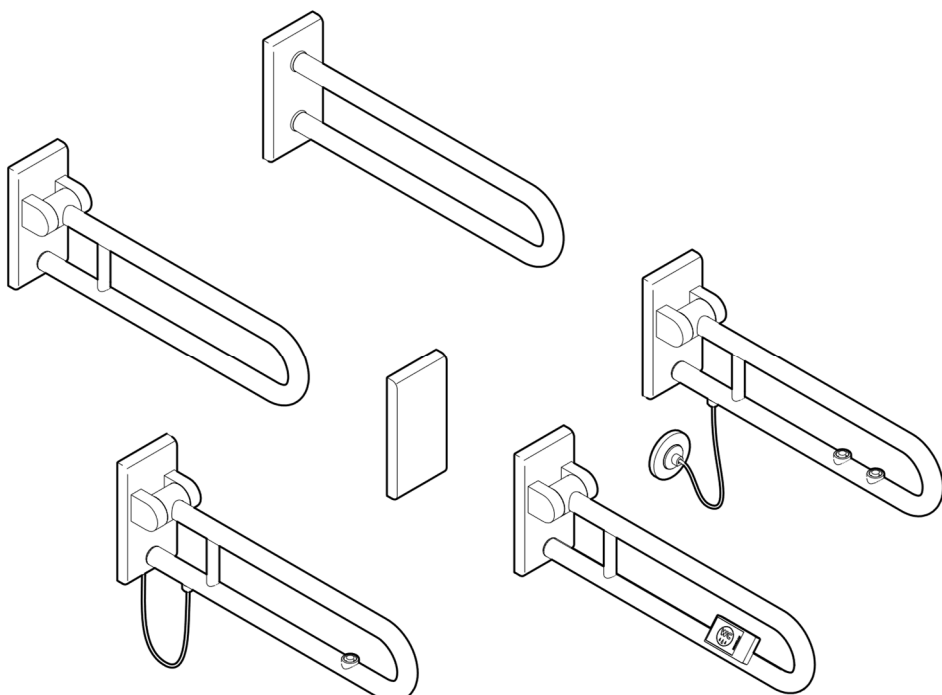
NORMBAU

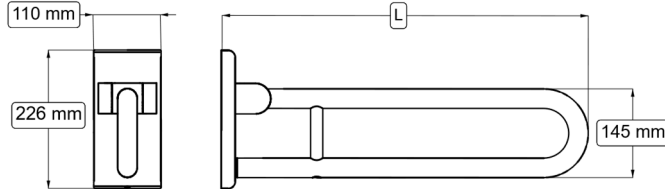
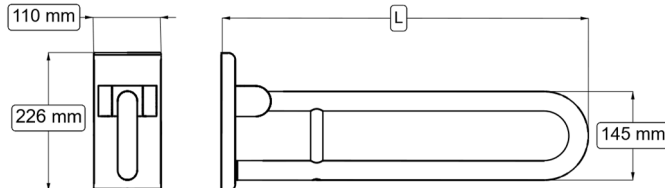
PASSION FOR CARE

Inox Care®

- DE** Montage- und
Gebrauchsanleitung
- EN** Fixing instruction and
directions for use
- FR** Instructions de montage et
d'utilisation
- NL** Montage- en
gebruiksaanwijzing

Weitere Informationen
Further information
Plus d'informations
Meer informatie
extranet.normbau.de

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Intended use

Lift-up support rails and wall support rails are intended as aids in the sanitary area. Technical handle systems offer support and stability and can help people with physical impairments to move independently and safely. In addition, lift-up support rails and wall support rails are designed to assist nursing staff and can also be used by healthy people.

Disclaimer of liability

NORMBAU assumes no liability for personal injury or property damage that arises or has arisen as a result of improper assembly, use or cleaning, in particular in disregard of the warning and safety instructions listed in these instructions.

Serious incidents that occur in connection with the product must be reported to the manufacturer and the competent authority of the country in which the user is established.

Warranty

The statutory warranty period is 2 years.

NORMBAU provides a total of 5 years warranty on functionality and stability.

If the intended operating conditions and the maintenance and cleaning guidelines are respected, the expected service life is at least 10 years. The product has been tested and validated based on this service life.

Warning notices

There is no toxic hazard from the material used by NORMBAU. Tested according to DIN EN ISO 10993-5:2009.

Risk of injury

The product surfaces can become very hot when exposed to external heat sources, e.g. direct sunlight or contact with hot water.

Risk of injury

If the max. loading permitted is exceeded, it can lead to permanent damage to the product. In this case, the product must be tested before further use!

Risk of injury

When folding down the seat, ensure that there is no part of the body under the product.

Safety instructions

Installation is to be carried out by a specialist and only with NORMBAU fixing sets or suitable fastening material. Before installation, the wall structure must be checked to determine the fastening material. Observe the information on the max. loading

permitted and the instructions for mounting! Following installation, the instructions must be made available to the user.

In case of questions, more information, uncertainties or if damage occurs, contact the NORMBAU support service +49 (0) 7843 704 0 or visit us at www.normbau.de.

Load

Max. loading permitted of listed products: 125kg

With max. length and max. loading permitted, the max. force per fastening is 3kN.

Maintenance and cleaning

NORMBAU products are not maintenance-free! The products are to be tested monthly by qualified specialists for proper function. In the event of defects (e.g. insufficient braking force), the assembly instructions must be observed.

Care and cleaning instructions

⚠ Risk of injury

Improper cleaning can cause damage to the surface and lead to injuries.

Stainless steel products from NORMBAU have a smooth, easy-to-clean surface. The material has proven itself both with frequent external loads and when used in a corrosive environment. However, it should be noted that stainless steel can also rust without care and by the action of aggressive agents. Damage to the passive protective layer created by oxygen must be avoided. Dirty parts can be cleaned with a soft, damp cloth or sponge. If heavily soiled, we recommend a commercially available detergent. Common wiping disinfectants can be used. Cleaners with hydrochloric acid, citric acid and acetic acid (tile cleaner) and cleaning sponges made of steel wool are not suitable for use on stainless steel.

More information about
maintenance and cleaning

Disposal

Disposal of the product and packaging is subject to the relevant national regulations and must be disposed of in the prescribed manner.

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GB Pre-wall installations with panels and integrated mounting plate made of hardwood.

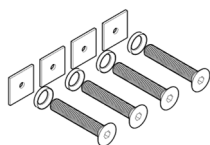
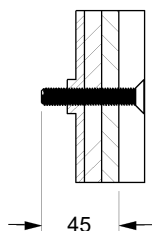
Lightweight walls, made of gypsum, flake board and hardboard with a backing made of at least 30 mm thick multi-layer glued wood.

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

GB Pre-wall installations with steel mounting plate.

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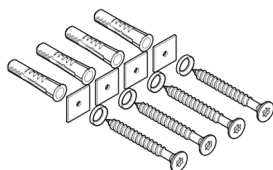
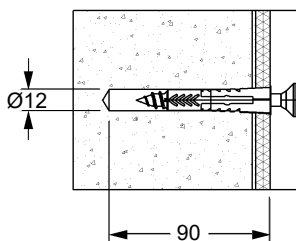
GB Masonry made of solid stone:
concrete $\geq$ B15, solid brick $\geq$ Mz 12,
sand-lime brick $\geq$ KS 12,
natural stone with compact grain

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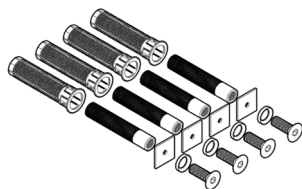
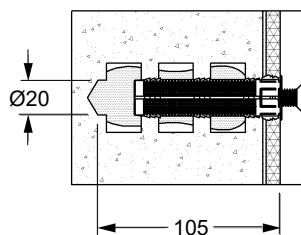
GB Masonry made of perforated stone:
perforated brick HLz $\geq$ 12, cavity brick KSL $\geq$ 12,
checker brick $\geq$ Hbl4, Hbn4, other hollow brick
materials, concrete / lightweight concrete,
solid brick $\geq$ Mz 12, sand-lime brick $\geq$ KS 12,
natural stone with compact grain, solid pumice

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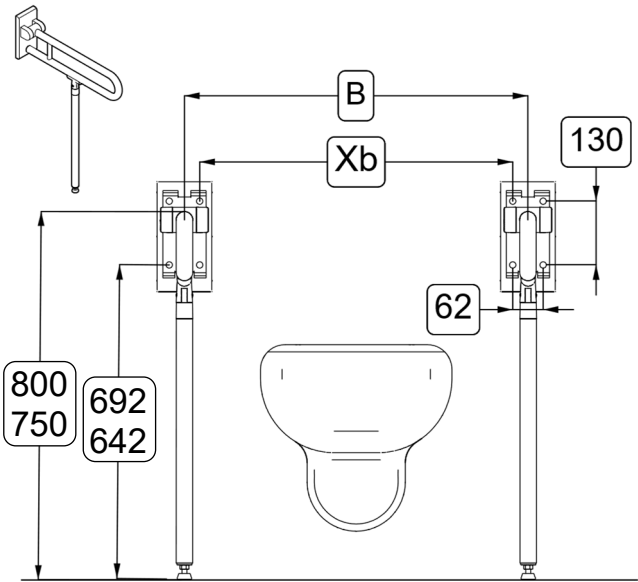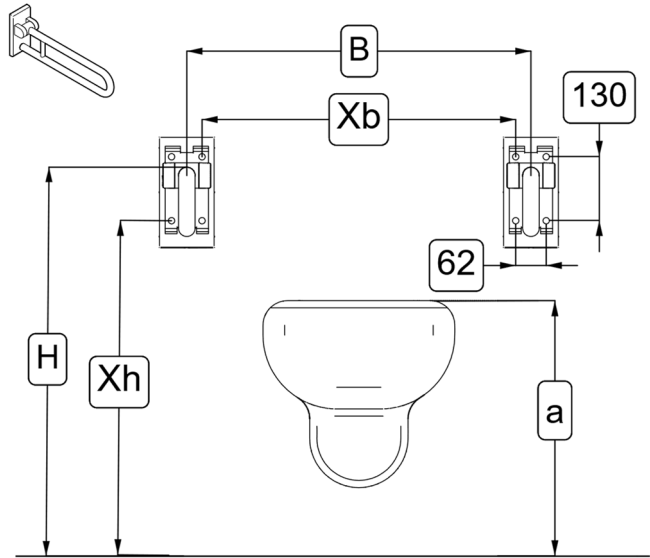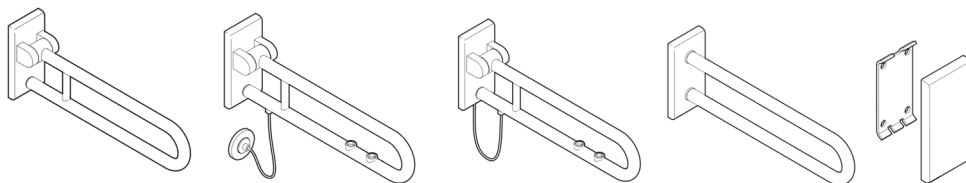
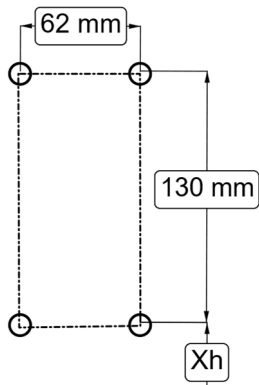
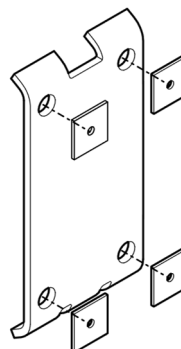
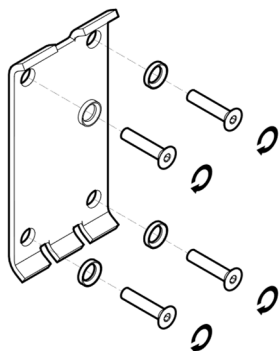
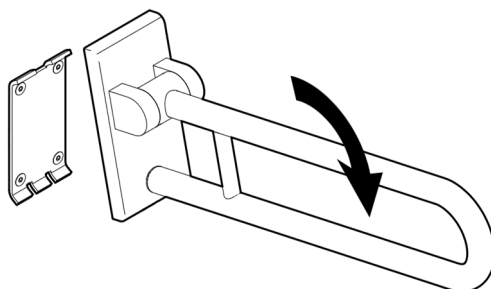
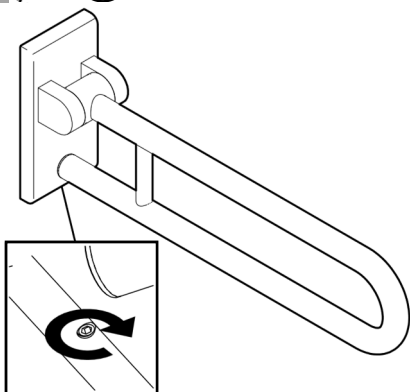
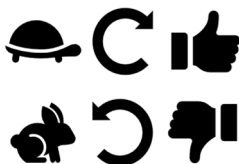
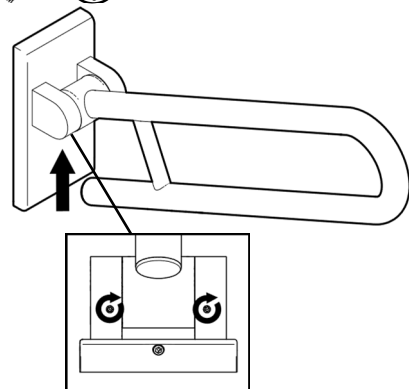
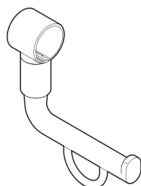
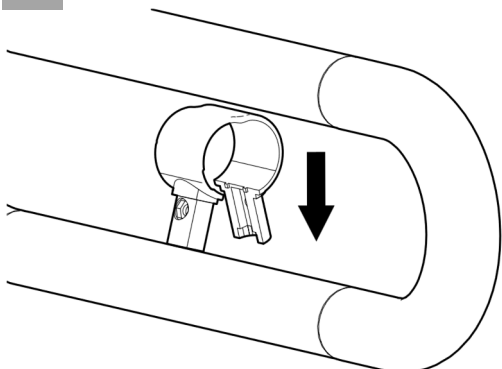
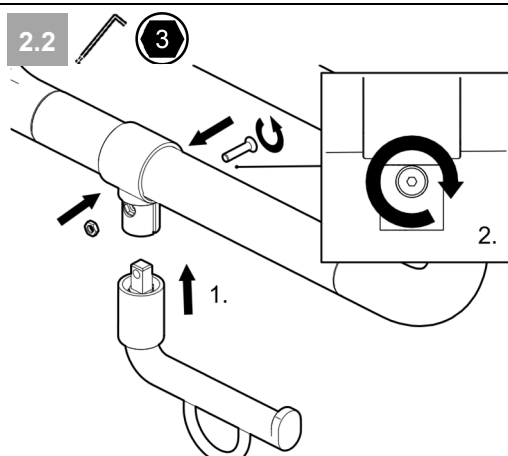
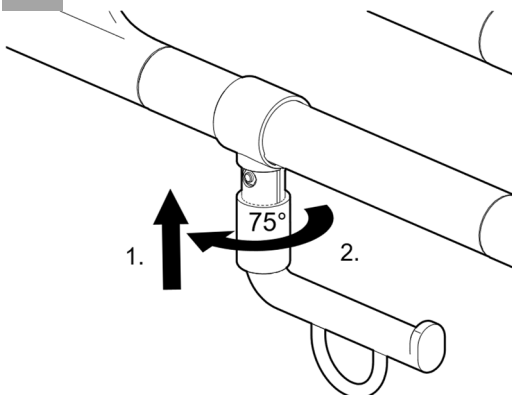
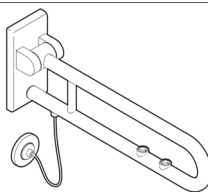
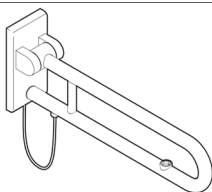
EN **⚠ Risk of injury due to electrocution**

Lift-up support rails with E-buttons must be installed outside the exposed water splash zone. Installation may only be carried out by qualified electricians, in compliance with the applicable regulations in accordance with VDE 0100 or the relevant national regulations.

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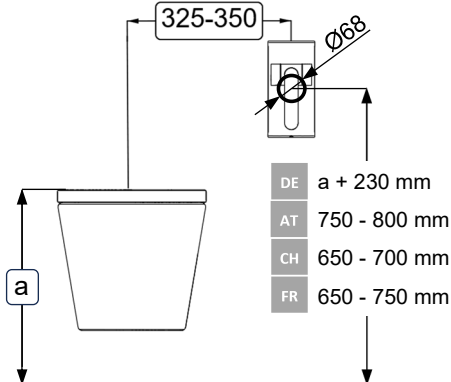
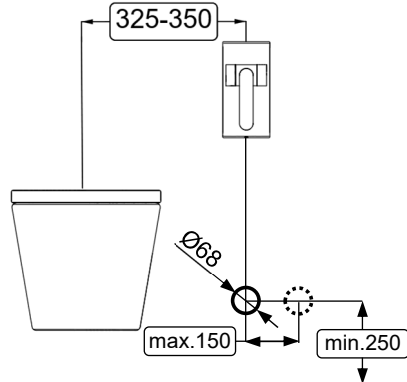
EN The hole diameter for the switch box is Ø68 mm.

When installing a transformer, a depth of at least 55 mm is recommended.

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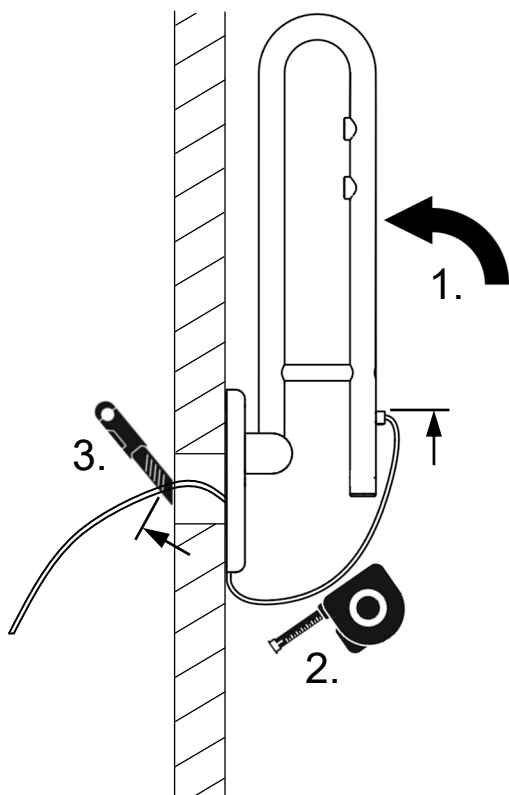
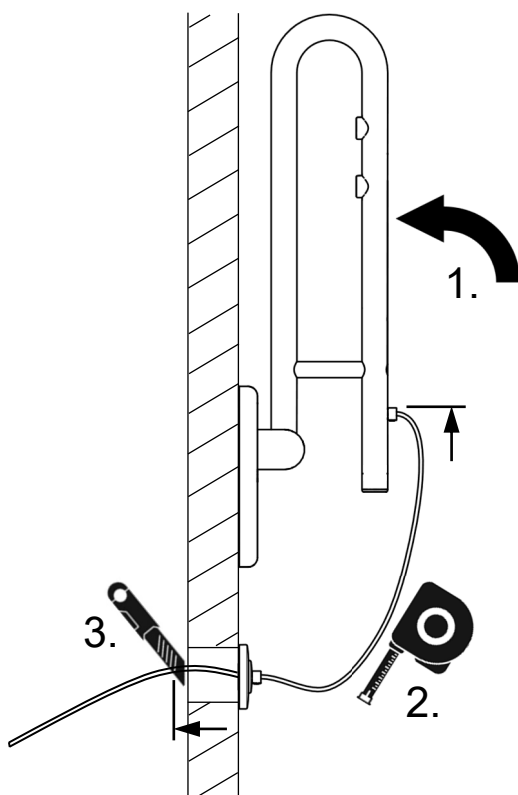
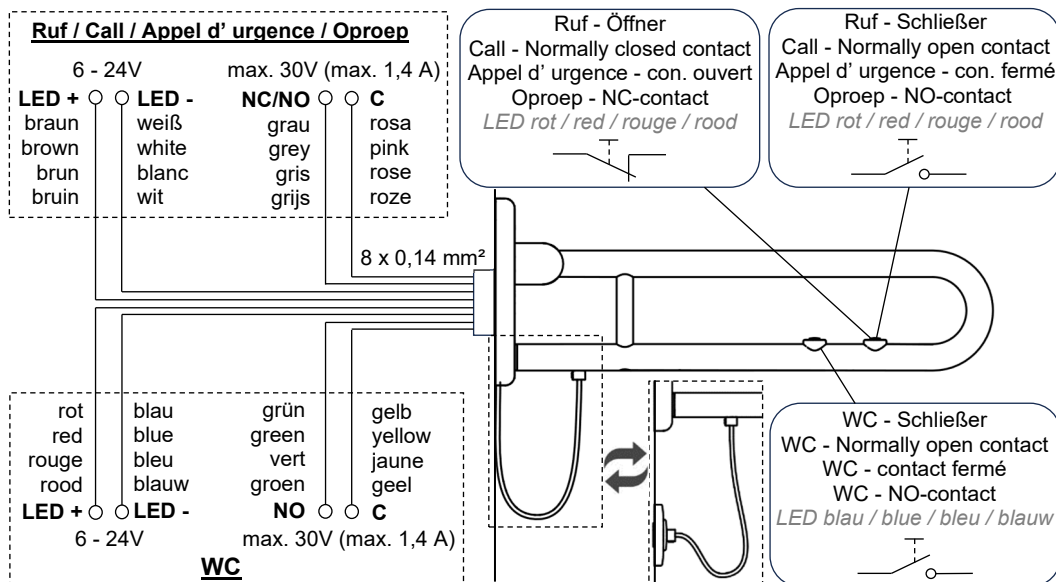
A maximum switching capacity of 30 V and 1,4 A safety extra-low voltage is permitted for WC controls or emergency call systems. If the e-switch is connected as a call button on an existing intercom according to VDE 0834 or the relevant national regulations, a cable relief must be fitted in the junction box. A minimum of protection type IP X4 must be provided for the junction box (junction box UP6).

A direct current voltage supply of between 6V and 24V is required to operate the LED lights.

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NORMBAU GmbH