
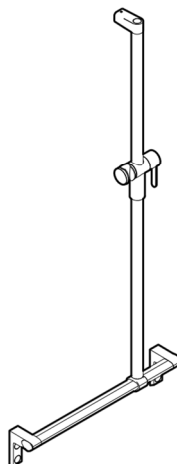
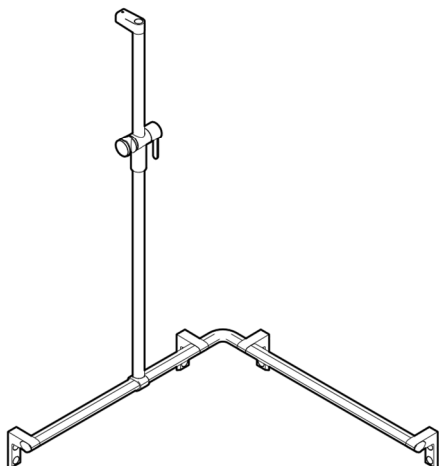
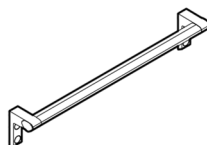
# NORMBAU


PASSION FOR CARE

**Cavere<sup>®</sup> Care**  
**Cavere<sup>®</sup> Care Chrome**


- DE Montage- und  
Gebrauchsanleitung
- EN Fixing instruction and  
directions for use
- FR Instructions de montage et  
d'utilisation
- NL Montage- en  
gebruiksaanwijzing

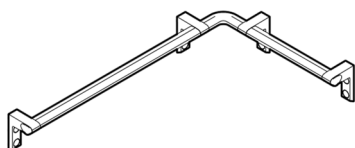
Weitere Informationen  
Further information  
Plus d'informations  
Meer informatie  
[extranet.normbau.de](http://extranet.normbau.de)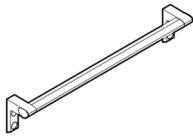


7484 060  
7484 090  
9484 060  
9484 090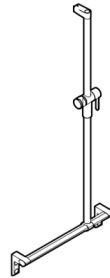

7485 051  
7485 052  
7485 060  
7485 090  
9485 051  
9485 052  
9485 060  
9485 090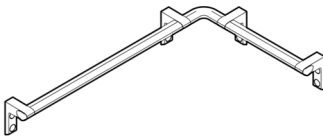

7484 044  
7484 045  
7484 046  
7484 075  
9484 044  
9484 045  
9484 046  
9484 075

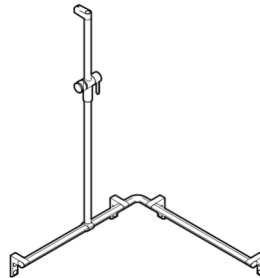7486 076  
7486 077  
7486 075  
7486 074  
7486 111  
7486 112  
7486 110  
7486 109  
7486 105  
7486 106  
7486 107  
7486 108  
9486 076  
9486 077  
9486 075  
9486 074  
9486 111  
9486 112  
9486 110  
9486 109  
9486 105  
9486 106  
9486 107  
9486 108

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### **Intended use**

Grab bars and grab rails as well as shower handrail systems are suitable as aids in the bathroom area. The technical handle systems offer support and holding options and support independent and safe movements for people with physical disabilities.

### **Disclaimer of liability**

NORMBAU assumes no liability for personal injury or property damage that arises or has arisen as a result of improper assembly, use or cleaning, in particular in disregard of the warning and safety instructions listed in these instructions.

Serious incidents that occur in connection with the product must be reported to the manufacturer and the competent authority of the country in which the user is established.

### **Warranty**

The warranty period is 2 years in accordance with current regulations.

If the intended operating conditions and the maintenance and cleaning guidelines are respected, the expected service life is at least 10 years. The product has been tested and validated based on this service life.

### **Warning notices**

There is no toxic hazard from the material used by NORMBAU. Tested according to DIN EN ISO 10993-5:2009.

#### **Risk of injury**

The product surfaces can become very hot when exposed to external heat sources, e.g. direct sunlight or contact with hot water.

#### **Risk of injury**

If the max. loading permitted is exceeded, it can lead to permanent damage to the product. In this case, the product must be tested before further use!

### **Safety instructions**

Installation is to be carried out by a specialist and only with NORMBAU mounting sets or suitable fastening material. Before installation, the wall structure must be checked to determine the fastening material. Observe the information on the max. loading permitted and the instructions for mounting! Following installation, the instructions must be made available to the user.

In case of questions, more information, uncertainties or if damage occurs, contact the NORMBAU support service +49 (0) 7843 704 0 or visit us at [www.normbau.de](http://www.normbau.de).

## Load

Max. loading permitted of listed products: 150kg

## Maintenance

NORMBAU products are maintenance-free.

## Maintenance and cleaning

### Risk of injury

Improper cleaning can cause damage to the surface and lead to injuries.

### Cavere Care

Powder-coated products from NORMBAU have a smooth, easy-to-clean surface. In order to maintain the decorative design and color of the coating of our products for years, cleaning and care must be carried out professionally. Slightly adhering dirt can be removed with a sponge and water to which a neutral cleaning agent (e.g. dishwashing liquid) is added. Immediately after each cleaning process, rinse with pure, cold water. A chamois or microfiber cloth removes the remaining wiping marks or drops of water. Abrasive or strong acidic/alkaline cleaning agents should never be used. With metallic coatings, heavy soiling should be cleaned as soon as possible, since dried and old dirt can often only be removed abrasively and with the risk of damage to the powder coating or a change in color. Common wiping disinfectants can be used.

### Cavere Chrome Care

Abrasive sponges and scouring agents should not be used to clean our Cavere Chrome products. We also advise against the use of cleaners containing solvent or acid, limescale removers, vinegar and cleaning agents containing acetic acid. The chrome surface can be cleaned using a little soap on a damp cloth. Limescale stains can be avoided by drying the tap after every use. If limescale deposits still build up, you can remove these using an environmentally friendly cleaning agent. The appropriate cleaning agent should only be left on for a short time before being rinsed thoroughly with water. In areas with hard water, it may be necessary to clean the chrome surface at specific intervals. Please observe the following points when cleaning:

- Residues from toiletries such as shampoo, shower gel or liquid soap should be removed quickly as these can leave permanent stains.
- Harsh cleaning methods and aggressive cleaning agents can damage surfaces and even penetrate the inside of the products.
- Do not apply spray cleaners directly to the chrome surface. Spray the cleaning agent onto a soft cloth and use this to wipe the surface instead.
- Use clean water to rinse the chrome surface thoroughly after cleaning.

- Do not use any cleaning agents that contain hydrochloric acids, formic acids, chlorine bleaching or acetic acids, as these can cause considerable damage.
- Be sure to treat limescale stains and other types of soiling correctly. Any damage caused as a result of improper treatment shall not be covered under warranty.

More information about  
maintenance and cleaning

### **Disposal**

Disposal of the product and packaging is subject to the relevant national regulations and must be disposed of in the prescribed manner.

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