
NORMBAU


PASSION FOR CARE

Inox Care®


- DE Montage- und
Gebrauchsanleitung
- EN Fixing instruction and
directions for use
- FR Instructions de montage et
d'utilisation
- NL Montage- en
gebruiksaanwijzing

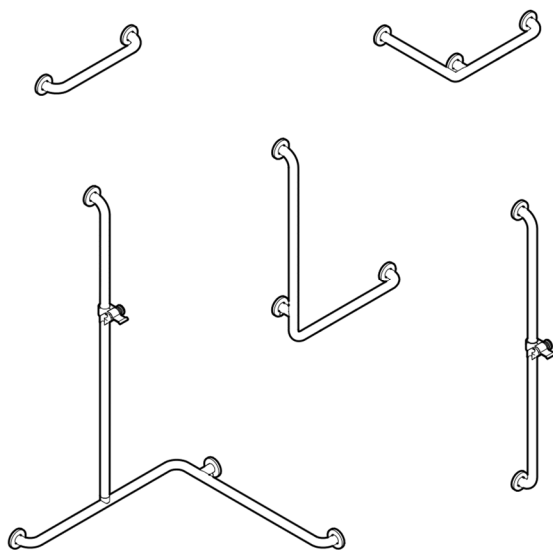
Weitere Informationen
Further information
Plus d'informations
Meer informatie
extranet.normbau.de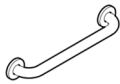

2053 030
 2053 040
 2053 050
 2053 060
 2053 080
 2053 100
 2053 110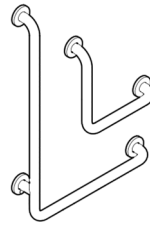

2054 020
 2054 025
 2054 030
 2054 041
 2054 130
 2054 141

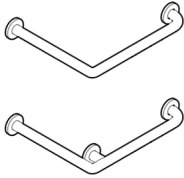
2054 050
 2054 060
 2054 160


2055 050
 2055 150


2058 010
 2058 030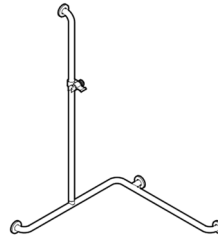

2062 210
 2062 230
 2062 240
 2062 310
 2062 330
 2062 340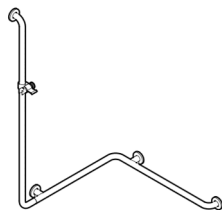


2061 220
 2061 230
 2061 240
 2061 250
 2061 320
 2061 330
 2061 340
 2061 350

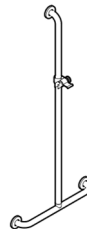2062 410

2053 160

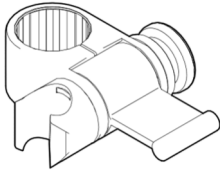0515 100

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Intended use

Grab bars and grab rails as well as shower handrail systems are suitable as aids in the bathroom area. The technical handle systems offer support and holding options and support independent and safe movements for people with physical disabilities.

Disclaimer of liability

NORMBAU assumes no liability for personal injury or property damage that arises or has arisen as a result of improper assembly, use or cleaning, in particular in disregard of the warning and safety instructions listed in these instructions.

Serious incidents that occur in connection with the product must be reported to the manufacturer and the competent authority of the country in which the user is established.

Warranty

The warranty period is 2 years in accordance with current regulations.

If the intended operating conditions and the maintenance and cleaning guidelines are respected, the expected service life is at least 10 years. The product has been tested and validated based on this service life.

Warning notices

There is no toxic hazard from the material used by NORMBAU. Tested according to DIN EN ISO 10993-5:2009.

Risk of injury

The product surfaces can become very hot when exposed to external heat sources, e.g. direct sunlight or contact with hot water.

Risk of injury

If the max. loading permitted is exceeded, it can lead to permanent damage to the product. In this case, the product must be tested before further use!

Safety instructions

Installation is to be carried out by a specialist and only with NORMBAU fixing sets or suitable fastening material. Before installation, the wall structure must be checked to determine the fastening material. Observe the information on the max. loading permitted and the instructions for mounting! Following installation, the instructions must be made available to the user.

In case of questions, more information, uncertainties or if damage occurs, contact the NORMBAU support service +49 (0) 7843 704 0 or visit us at www.normbau.de.

Load

Max. loading permitted for the products listed below: 175kg

2053 030	2054 030	2055 150	2061 320	2062 310
2053 040	2054 041	2058 010	2061 330	2062 330
2053 050	2054 130	2058 030	2061 340	2062 340
2053 060	2054 141	2061 220	2061 350	2062 410
2053 080	2054 060	2061 230	2062 210	
2053 100	2054 160	2061 240	2062 230	
2053 110	2055 050	2061 250	2062 240	

Max. loading permitted for the products listed below: 125kg

2053 160	2054 020	2054 025	2054050
----------	----------	----------	---------

Maintenance

NORMBAU products are maintenance-free.

Maintenance and cleaning

Risk of injury

Improper cleaning can cause damage to the surface and lead to injuries.

Stainless steel products from NORMBAU have a smooth, satin finish, easy-to-clean surface. The material has proven itself both with frequent external loads and when used in a corrosive environment. However, it should be noted that stainless steel can also rust without care and by the action of aggressive agents. Damage to the passive protective layer created by oxygen must be avoided. Dirty parts can be cleaned with a soft, damp cloth or sponge. If heavily soiled, we recommend a commercially available detergent. Common wiping disinfectants can be used. Cleaners with hydrochloric acid, citric acid and acetic acid (tile cleaner) and cleaning sponges made of steel wool are unsuitable for use on stainless steel.

More information about
maintenance and cleaning

Disposal

Disposal of the product and packaging is subject to the relevant national regulations and must be disposed of in the prescribed manner.

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NORMBAU GmbH